



VEMO

Continuous ECG Recorder

- Quick Guide -

This is veterinary use only.
Please refer to the operation manual before use.

vemo@bionet.com
Bionet Co., Ltd. - 5F, Dreammark1 Data Center 61 Digital-ro 31 gil, Guro-gu, SEOUL, 08375
+82-2-6300-6410

support@bionetus.com
Bionet America Inc. - 2691 Dow Ave, Suite B, Tustin, CA 92780
+1-877-924-6638

System component



Charging cable



Charging cradle




VEMO ECG sensor



Hydrogel Adhesive 2packs
(1pack / 4pcs)

App store : Bionet VEMO

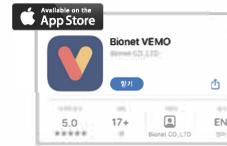

Charging VEMO

Please check!



1. Make sure two charging points are connected.



2. Place the VEMO sensor in the charging cradle.



3. Close the cradle.

4. Connect the USB cable to charging cradle.



How to Authenticate VEMO

How to authenticate VEMO

When downloading the VEMO app from the app store, it is set up to process authentication. To use the VEMO app, the wireless, wearable sensor is used for authentication.



1. After downloading the app, please input the hospital's information.



2. Place the wireless sensor on the tablet's screen for the authentication process.




3. After authentication is completed, go to the main screen like the picture on the left.



4. If the authentication was not completed, the app cannot be used & an error message will show up like the picture on the left. (press the back button to restart the authentication process)

Steps for pairing


* Caution - Several VEMO sensors around the tablet or mobile can cause pairing error.

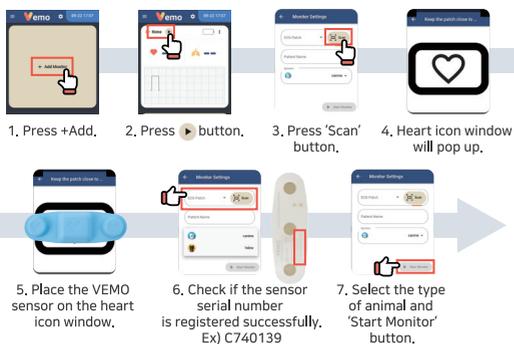

When 'Auto Scan' has a problem (manual pairing)




Steps for applying adhesive to VEMO sensor

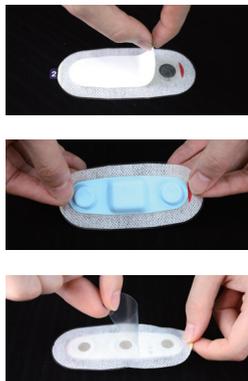

1. Peel out adhesive No.1.
2. Attach the VEMO sensor on the No.1 side.
3. Peel out adhesive No.2.



4. Placed the VEMO sensor on the heart with its heart icon 90 degree vertical and clockwise.

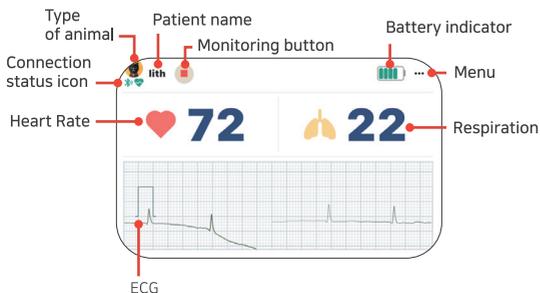

Status icon

- Pairing status




Connected Disconnected Standby Error

- ECG lead status



Connected Error or sensor not applied

Chart review

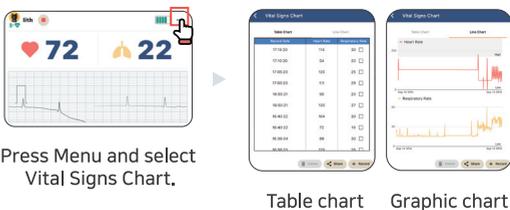


Press Menu and select Vital Signs Chart.

Table chart

Graphic chart

Sharing Chart

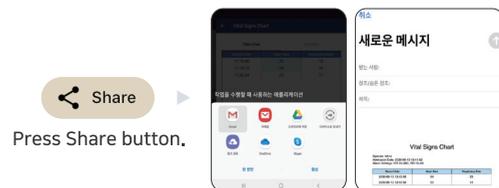

Press Share button.

Select sharing method to convert the chart into PDF format.



1. The recommended pairing range between VEMO sensor and tablet is 3m(10ft) up to 10m(32ft).



2. Use VEMO genuine hydrogel adhesive patch to protect VEMO sensor's ECG lead. In case of use other adhesive, warranty will be voided.



3. VEMO sensor is sensitive to heat, magnetic fields, and metal objects. Make sure VEMO sensor is not located near those materials.



4. Check the sensor serial number is read & registered successfully.



5. When VEMO is in use, check pairing for ECG lead status and battery indicator.



Hello,
I'm VEMO!



We deliver highest quality
of connected healthcare solutions
through constant innovation and sharing
the value with our customers.