



Portable X-ray system

PORT-X IV

For Veterinary Use Only

User Manual

UM-206 CE VET

2022.06.30 Document Ver. 1.5

About the Manual

The following manual for the PORT-X IV covers all operational instructions and pertinent information.

Please read through this manual for proper and safe use of the equipment.

GENORAY Co., Ltd. is continuously developing the equipment, so it's manual and configuration may change without the user's prior consent. If you have any questions about the equipment or manual, please contact our service center.

The manufacturer and importer are responsible for ensuring the safety and reliability of the equipment along with its installation, calibration and repair by qualified personnel.

Manual Conventions

This manual uses the following conventions:



Emphasizes important information.



Indicates important instructions.

If not observed, malfunction or damage to the Product or other property may occur.



Indicates warnings and safety instructions

If not adhered to, there is a serious risk of injury to be patient and/or the operator

Used Symbols

Symbol	Description	Location
	AC (Current)	
	Protective earth (Ground)	Boards
	"ON" (Power)	ON/OFF Switch
	"OFF" (Power)	ON/OFF Switch
	X-Ray generator, X-Ray inspection	Label
	Warning, electricity	Generator
	WEEE Mark	Label
	Type-B Applied Part	Label
	Name of Manufacturer and Address	Label
	Manufactured Date	Label
	Equipment Serial Number	Label
	Refer to the User Manual	Label
	The CE symbol indicates that this product complies with the European Directive for Medical Devices 93/42/ EEC as amended by 2007/47/EC as a class IIb device.	Label Manual
	Authorized Representative in the European community	Label Manual

Table of Contents

About the Manual	i
Table of Contents	iii
1. Equipment Introduction and Safety Information	7
1.1 Equipment Introduction	7
1.2 Precautions before use.....	9
1.3 Suitability	10
1.4 Precautions During Use	10
1.5 Storage Precautions	11
1.6 Electrical Safety and Fire	12
1.7 Significant zones of occupancy	13
1.8 Radiation Safety	14
1.8.1 Protection area of backscatter radiation	15
1.9 Equipment Overheating.....	18
1.10 Usage Environment.....	19
1.11 Waste Treatment Methods.....	20
1.12 Precautions when using a wireless USB Dongle.....	20
2. Composition of PORT-X IV	21
2.1 Structure and Naming	21
2.2 Buttons for Equipment Operation.....	23
2.3 Exposure Restriction Mechanism	26
2.4 Connector	27
2.5 Battery.....	29
2.5.1 Coin cell.....	31

2.6	Display	32
2.6.1	Menu.....	34
3.	Directions	43
3.1	Preparation before use.....	43
3.2	Image Acquisition.....	44
3.2.1	Filming Method - Not Using the Sensor	44
3.2.2	Filming Method - Using the Sensor	46
3.3	Image Transfer.....	50
3.3.1	Automatic Transfer	50
3.3.2	Manual Transfer.....	51
3.4	Image Management	52
4.	Precautions after Use	57
4.1	Regular Inspection	57
4.2	Care and Cleaning	57
5.	Equipment Specifications	59
5.1	System	59
5.2	X-ray Generator.....	60
5.3	X-ray Tube	61
5.4	Battery	64
5.5	Recharge Unit	64
5.6	Geometry and Mechanical Data	64
5.7	Sensor (Option).....	65
5.8	Dimensions and weight.....	66
5.9	Equipment Labeling	67
Appendix1.	Error Messages and Interim Measures	70

A.1.1 Error message	70
A.1.2 Warning Message	72
Appendix2. X-ray Exposure settings.....	74
A.2.1 Initial setting	74
Appendix3. Air-kerma & Dose Information	75
Appendix4. Maintenance.....	76
A.4.1 Cleaning.....	76
A.4.2 Dose Confirm	77
A.4.3 Check image	79
A.4.4 Service.....	81
A.4.5 Returns and repairs	81
A.4.6 Maintenance table	82

This page is intentionally left blank.

1. Equipment Introduction and Safety Information

1.1 Equipment Introduction

The diagnostic x-ray system PORT-X IV is a device for photographing the teeth and bones, etc. of patients for accurate diagnosis. It is designed to realize high quality images and reduce unnecessary radiation exposure, and is a portable device.

The multiple name of the PORT-X IV is ZEN-PX4.

- Intended Use
 - The PORT-X IV is for Veterinary use only.
- Features of the PORT-X IV
 - Convenient User Interface
 - Possible to set optimal x-ray photography time for each site
 - Using IPS LCD well-equipped for viewing angles
 - Reduced overall weight to improve portability
 - When connected with a Genoray sensor, it is possible to view the images obtained by equipment on the back LCD



This equipment conforms to Class II according to the Rule 10 of Classification Criteria, Annex IX of Council Directive 93/42/EEC.

-
- Firmware version
 - Version 1.0.6.12



The Firmware is constantly updated so the manual version and actual equipment may differ in versions.

To set automatic firmware updates, please refer to "2.6.1 Settings – Menu."

- Equipment Warranty

All equipment of GENORAY Co., Ltd. receives warranty repairs free of charge for breakdowns and defects occurring during warranty period after purchase.

In the following cases, a service fee will be charged after the repair:

- The warranty period has expired.
- Breakdown was caused by a natural disaster such as earthquake, fire, lightning, etc.
- Breakdown due to improper transportation of equipment or careless use.
- Breakdown was caused by repair or modification by someone other than Genoray Co., Ltd. or designated service personnel.



We will make available on request circuit diagrams, component part lists or other information that will assist service personnel to repair those parts of PORT-X IV.

1.2 Precautions before use

Users are responsible for equipment operation and maintenance. For your safety, please be familiar with the following precautions and perform regular inspections.

- There is always potential risk present with the use of medical devices that use X-rays and electricity.
- Please read the safety related information provided in the manual such as the method of use and methods for handling emergency situations, etc.
- Please read this manual thoroughly before using this equipment.
- This equipment may only be used by legally qualified doctors, dentists and radiologists.
- Please check condition of contact, polarity and confirm the equipment is operating correctly.
- Please check that all cord connections are correct and complete.
- Since a rechargeable battery is used, please check the charge before use and if the charge is insufficient, use the dedicated adapter to charge it.
- If any abnormality is found or if the battery needs to be replaced, consult the distributor or manufacturer.
- Only use the adaptor and battery supplied by Genoray Co., Ltd.
- Charge the device at a safe place at room temperature.
- In case of malfunction, do not attempt to operate the system until it has been inspected by a qualified service technician.
- Instruct the patient to remove any kind of accessories and metals, before the imaging. There may be shadows and reflections when you take an image with the abovementioned objects. For safety reasons, it is recommended that anything that goes around the neck, be removed.
- Do not change this equipment without the permission of the manufacturer.



Do not modify this device without the permission of the manufacturer.



This equipment is legal qualified doctors, dentists, radiologists can only be used.

1.3 Suitability

The user of this equipment is responsible for verifying the X-ray exposure, leakage dose, valid beam center and accuracy of kV and mAs.

- This equipment may be used only by legally qualified doctors and radiologists.
- Only authorized persons may enter the X-ray room.

1.4 Precautions During Use

- Use it by following user's age, gender, body conditions and be aware to not exceed the time required for diagnosis and treatment.
- Use as directed by a specialist. It is to be used only by authorized personnel.
- Continue to monitor the device as a whole and the patient for any abnormalities.
- If abnormalities are found in the device as a whole or in the patient, take appropriate measures, such as keeping the patient in a safe condition and stopping the operation of the device, etc.
- In the event of a malfunction of the device, do not touch the device but check the power supply and for the appearance of abnormality based on the judgment of the person in charge. Make a request to the manufacturer's or seller's specialist for repair after marking it appropriately.
- The Micro SD card should be examined on a regular basis, and if it is unreadable, you must purchase a new SD card.
- We recommend only using SD cards that the manufacturer has authorized or provided.
- When removing the SD card from the equipment, make sure to turn the power off.
- If you attempt to remove the Micro SD card while the power is on, it may cause data loss or damage to the Micro SD card and cause errors in the card.
- Do not disassemble or modify the Micro SD card.
- Do not disconnect the sensor from the equipment during the X-ray exposure.

Disconnection of the sensor during the X-ray exposure can cause damage to or missing of the data, or cause damage to the equipment.

- Do not modify or disassemble the sensor arbitrarily.



CAUTION

Fix the tripod when using the device.

1.5 Storage Precautions

- Please store it at a place away from water.
- Store in a place where adverse effects are not caused by atmospheric pressure, temperature, humidity, ventilation, consistency, dust, salt, ions, etc.
- Pay attention to safety conditions such as slope, vibration, shock, etc.
- Do not store in chemical storage or gas generating locations.
- When using the device, do not use any device that can generate external electromagnetic fields, such as a cell phone.
- Regularly check the performance of the device and components.
- When using a device that has not been used for a long time, check with the manufacturer to make sure that it is operating completely normally.
- Use a neutral detergent to clean regularly and ensure that liquid does not go into the equipment.
- The following disinfectants are recommended for surface disinfection
 - Incidin(3% in water)
 - Ultrasol F(5:1 in water)
- Do not use corrosive detergents or disinfectants.
- Do not keep a fully charged device unused for long.
- For storage, charge the battery as much as 65-75% of its capacity every six months.
- Use or storage of the device's battery in an environment hotter than 60°C can cause a damage to it.

1.6 Electrical Safety and Fire

Please check the following before using this equipment.

- This equipment is powered by a battery.
- While charging the battery, please check if the plug supplying power is appropriate.
- The same should be checked for the power input of all peripheral devices (signal input/output including peripheral devices).



PORT-X IV is a Class I device and Type B device with respect to its rated safety type and grade.

This equipment is not protected against liquid.



To avoid electrical interference when charging the power supply of this equipment, keep it away from power generators, other X-ray equipment, and broadcasting stations. If this equipment shares power with other electrical or electronic equipment, abnormal images may occur.



The electrical circuits in this equipment utilize voltages that can cause serious injury or death from electric shock. To avoid such risks, users should never remove the cover from any part of the unit.

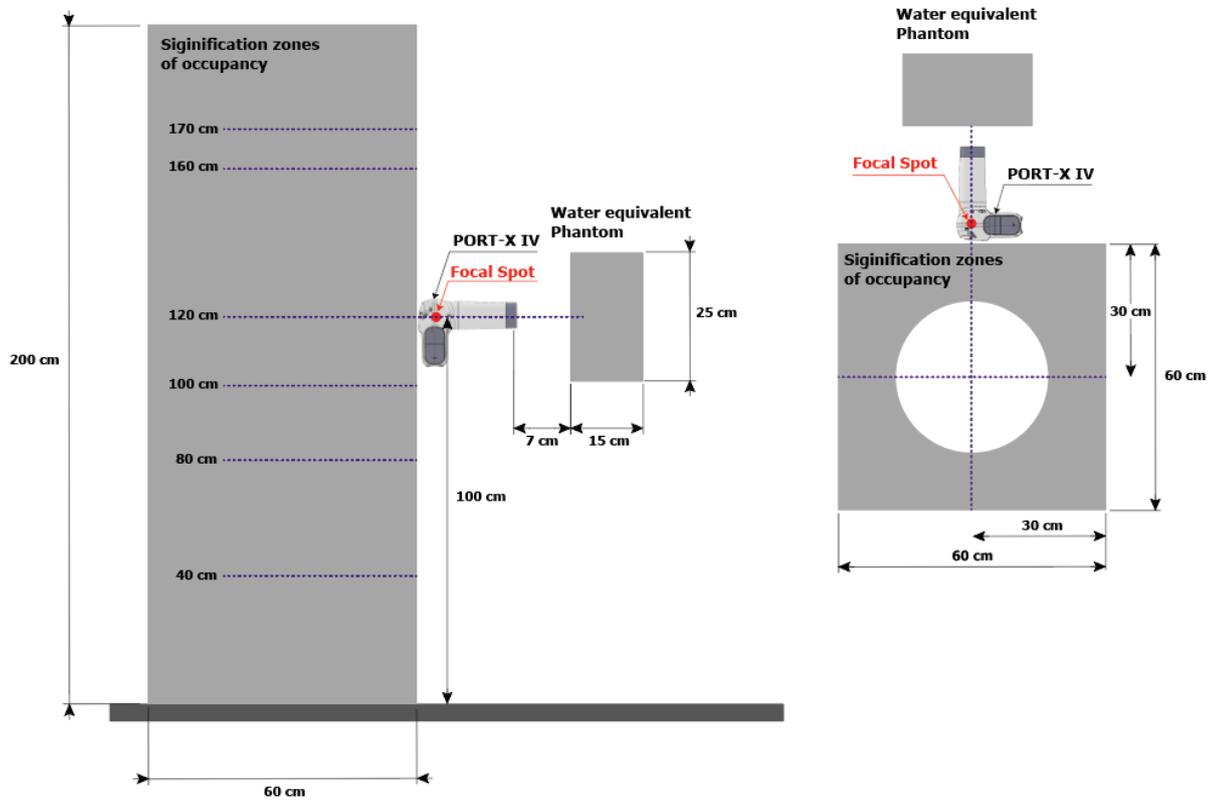


To charge the battery, use the provided charger. If it is used for any purpose other than charging this equipment, it may cause damage or breakdown of the equipment.



This system is not waterproof. Exposing to any liquids including water and soap could cause electrical short circuits leading to electric shock or fire hazards. In case liquid does spill on the unit, DO NOT connect power cord or turn the system on until the liquids have dried or evaporated completely.

1.7 Significant zones of occupancy



Height(cm)	Highest permissible Air kerma In one year, mGy	Measured stray radiation in one hour mGy
0	1.5	0.101
20	0.7	0.309
40	0.7	0.334
60	0.7	0.140
80	0.7	0.081
100	0.7	0.149
120	0.7	0.242
140	0.7	0.221
160	0.7	0.208
170	0.7	0.108
180	0.7	0.198
200	0.7	0.129

1.8 Radiation Safety

Must comply with all radiation laws and regulations with the local jurisdiction.

- When taking exposures, the x-ray room of the floor, wall material and radiation protection must be strengthened.
- When taking exposures, operators and service personnel must protect themselves from radiation.
- Protect the patient from scattered radiation by placing a protective lead apron over the patient.
- When taking an X-ray exposure of a child or pregnant woman, consult a doctor.
- Before exposing the X-ray, ask any female patient of childbearing age whether she might be pregnant.
- All patients must be provided with a shielded apron for gonad protection.
- We recommend you use Backscatter shield to prevent unnecessary exposure.
- Always be careful not to push the exposure button unintentionally.
- Please refer to the following for radiation safety information for pediatric.
 - Always comply with recommended X-ray dose when exposing the X-ray of a pediatric.
 - Avoid additional exposures by unnecessary movement of a pediatric, must be accompanied by a parents.
 - Always use thyroid collars.
 - Use special care when imaging patients outside the typical adult size range.



The unit may be dangerous to the user and the patient, if the safety regulations and standard procedures in this manual are ignored.



The unit can be used at all age that requires X-ray diagnosis, but consult a doctor if the system is used for using children and pregnant women.



The user should be operate this equipment in the area more than 2 meter with hand switch and tripod.

1.8.1 Protection area of backscatter radiation

During the X-ray exposure, the radiation scatters not only in the target direction, but also backscatter radiation.

To protect themselves from backscatter radiation, all users are recommended to use proper protective equipment, such as a backscatter shield, a lead apron, a thyroid guard, and etc.



Protection Area for the Target Position

The protection area can vary depending on the position of the user, the patient, and the equipment.

Refer to the following example to use the equipment in the optimal position.

- Correct position for exposure

In a correct position, the maximum area of protection against the backscatter radiation can be secured.

The user should observe the following position for protection against the radiation.

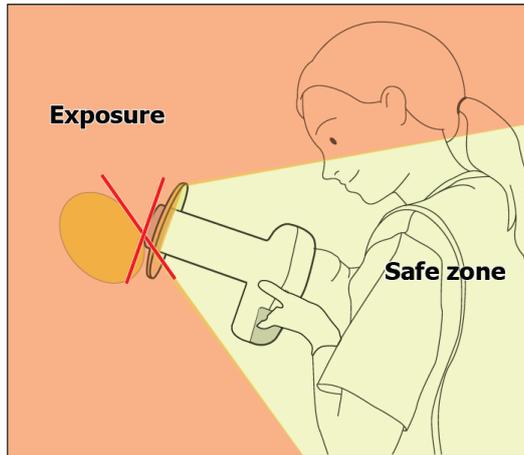
1. Place the target area of the patient parallel with the backscatter shield.
 2. Contact the tip of the cone tightly to the target area.
- ※ When necessary, tilt the head of the patient.



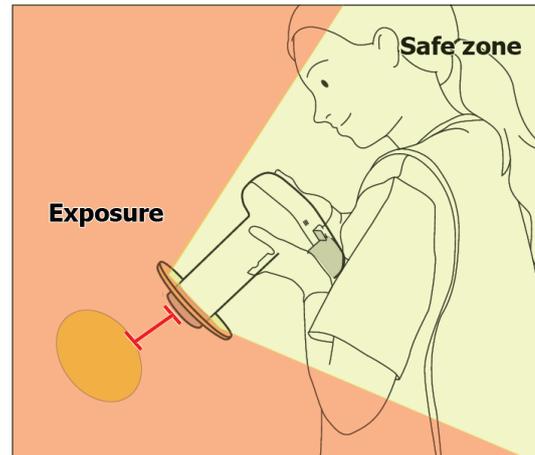
- Incorrect position for exposure

In an incorrect position, the protection area against the backscatter radiation becomes narrow and a part of the user can be exposed to the radiation.

- The target area of the patient is not parallel with the backscatter shield.
- The equipment is separated from the target area.
- To protect the user, it is recommended to use the equipment in a correct position.



Incorrect angle



Incorrect distance

Scatter Radiation Safety Instruction

- If the patient and assistant is required within 2 m(6 ft) of the exposure, the assistant should wear a proper set of protective gears as regulated by the law and instruction of the country and the region.
- If X-ray exposure is inevitable outside of the protection area against the backscatter radiation, try to protect the user as much as possible using protection such as a lead apron, a thyroid guard, etc.
- If the backscatter shield and the cone is damaged, do not operate the equipment.
- If the backscatter shield and the cone is damaged, stop using the equipment and call the manufacturer or the service manager.

1.9 Equipment Overheating

The below warning is displayed when the high voltage generating device is overheated.

Temperature	Color	X-Ray Exposure Status
Room temperature 5°C or below	 Grey	X-ray exposure impossible
Room temperature 6°C to 13°C	 Blue	X-ray exposure possible (recommended to use after warming up)
Room temperature 14°C to 40°C	 Green	X-Ray exposure possible
Room temperature 41°C to 44°C	 Yellow	X-ray exposure possible (recommended to use after cooling)
Room temperature 45°C or above	 Red	X-ray exposure impossible (overheating of high voltage generating device)

- In case of overheating of high voltage generating device, turn off the power and wait for 2 to 3 hours for cooling time. After that, turn on the power and check if exposure is possible.



If you use the equipment continuously without cooling time, the X-ray tube may be damaged. Therefore, use it after allowing it to have the post-exposure cooling time.

1.10 Usage Environment

- Allowable Environment
 - Temperature: 6 °C to 35 °C (42.8 °F to 95 °F)
 - Relative Humidity: 10% to 80%
 - Atmospheric pressure: 800 to 1060 hPa (80 to 106 kPa)
- Optimal temperature and humidity
 - Temperature: 19 °C to 25 °C (66.2 °F to 77 °F)
 - Relative Humidity: 40% to 60%
- Move & Storage Environment
 - Temperature: -10 to 60°C (14 °F to 140 °F)
 - Relative Humidity: 10 to 80%
 - Atmospheric pressure: 500 to 1060 hPa
- Avoid the following conditions when using and storing the equipment.
 - Exposure to direct sunlight
 - Exposure to dusty conditions
 - Exposure to high humidity
 - Environments that may cause problems with the ventilation of the equipment
 - Exposure to a salt-containing atmosphere
 - Exposure to chemical or hazardous gases

**WARNING**

Use of the device in an environment that does not meet the service environment may lead not only to the degradation of its life and performance, but also it can cause a fire or an explosion.

1.11 Waste Treatment Methods

Disposal of worn out equipment and accessory wastes is as follows.

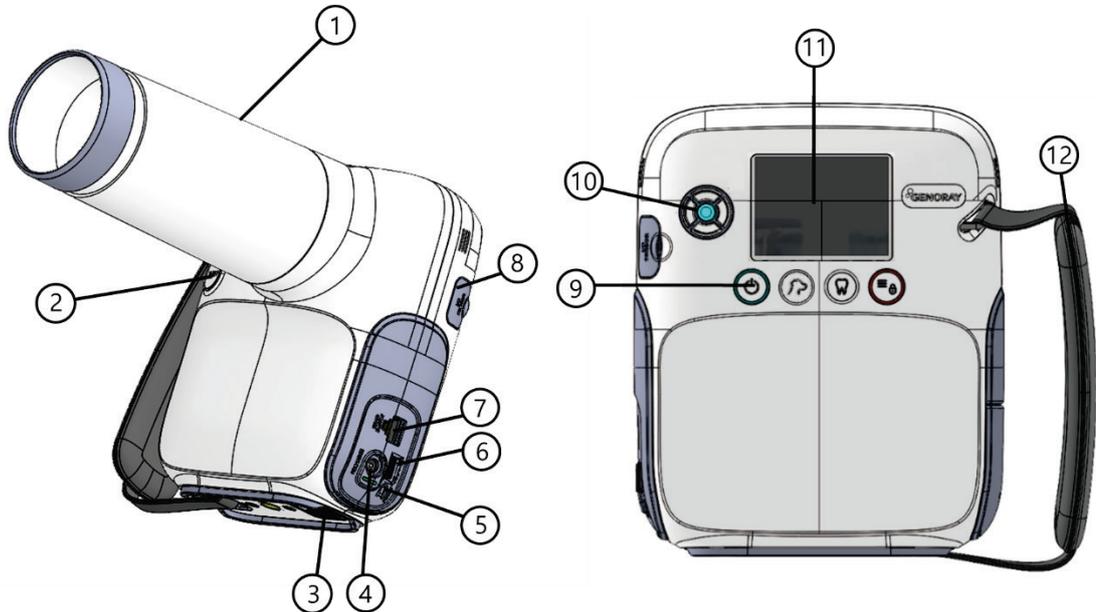
No.	Component	Disposal Method
1	X-ray Tube	Industrial Waste Treatment Facility
2	Plastic	Recycled
3	PCB & Harness or Other Processing Material	Industrial Waste Treatment Facility

1.12 Precautions when using a wireless USB Dongle

- We recommend that you only use products that have been provided by CE/FCC certified by Genoray.
- We recommend using a WPA-PSK or a higher form of security on your router, preventing external devices from connecting. (PORT-X IV cannot connect to routers without a password).
- Regularly change passwords.
- If the password is exposed externally, immediately change the password.
- Block any external devices connecting to the router that is being used.
- Have the PC OS system updated at all times.
- Use antivirus programs for your PC and keep the antivirus programs updated at all times.
- Have PORT-X IV updated to the newest Firmware
- Do not connect PORT-X IV with the PC if the PC is infected with a malicious virus.
- If the PC is infected by a malicious virus, immediately sever the network connection and seek security advice.

2. Composition of PORT-X IV

2.1 Structure and Naming



No.	Naming	No.	Naming
1	Exposure Restriction Mechanism	7	Hand switch connector
2	X-ray exposure button	8	Oral sensor USB connector
3	Wireless USB dongle connector	9	Power button/Status indicator LED
4	Charging connector	10	Move/Select
5	Reset button	11	Display
6	Micro SD card connector	12	Strap Handle

Component

Image	Description
	Battery
	Charger
	FFC jig
	Micro SD
	Hand switch
	Back Scatter Shield
	Wireless USB dongle
	Neck Strap
	Intra Oral Sensor (Option)
	Tripod

2.2 Buttons for Equipment Operation

Optimal images can be obtained by subdividing the patient's size and tooth position selection as well as equipment ON/OFF control.



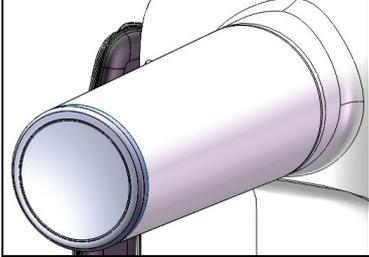
Button	Description				
	<p>[Move/Select]</p> <ul style="list-style-type: none"> There are two types to use the Move/Select function: trackball type and button type. The function works differently in each type. Please check the type of your equipment first and refer to the proper usage. <table border="1" data-bbox="502 1160 1264 1332"> <tr> <td data-bbox="502 1160 884 1279">  </td> <td data-bbox="890 1160 1264 1279">  </td> </tr> <tr> <td data-bbox="502 1288 884 1332">Trackball type</td> <td data-bbox="890 1288 1264 1332">Button type</td> </tr> </table>			Trackball type	Button type
					
Trackball type	Button type				
	<p>[Trackball]</p> <ul style="list-style-type: none"> Roll the ball in the middle of the device to move up, down, left, and right from the X-ray exposure setup, menu, keyboard, etc. Click the ball to select an item in the menu or the system. 				
	<p>[Button]</p> <ul style="list-style-type: none"> Move button (): moves up/down or right/left on the X-ray scan configuration, menu, keyboard, etc. Select button (): selects the item where the cursor is located under a menu or system. 				

Button	Description
	<p>[Equipment ON]</p> <ul style="list-style-type: none"> If you press the power button for at least 0.5 seconds, the status indicator button turns on blue with a start sound and the unit turns on. <p>[Equipment OFF]</p> <ul style="list-style-type: none"> If you press the power button for at least 2 seconds, the equipment turns off with a shutdown sound. The equipment is set to use power saving mode by default. <ul style="list-style-type: none"> Power Saving Mode: If the device is not used for a set period of time, it will change to the power saving mode. In power saving mode, a white LED lights up on the trackball() or select button. () Shut down: While in power saving mode, if you do not use the device for a specified period of time, <ul style="list-style-type: none"> It will shut down automatically. If the battery has less than 10% remaining, the device will shut down with a shut down sound. Press one of the buttons to cancel the power saving mode. Power saving time and auto shutdown time can be changed from "Menu → Settings → Set Time."
	<p>[Changing Patient Size]</p> <ul style="list-style-type: none"> If you press the patient size button, the patient size changes. The x-ray exposure time changes depending on the size of the patient. Patient size is categorized as "large, medium and small."
	<p>[Tooth Selection]</p> <ul style="list-style-type: none"> Select the tooth position. Use the trackball() or move button () to select the maxillary/mandibular and the tooth to be imaged. Depending on the tooth, the x-ray exposure time will vary.
	<p>[Menu]</p> <ul style="list-style-type: none"> The menu window appears or disappears. <p>[X-ray LOCK]</p> <ul style="list-style-type: none"> When pressed for 2 seconds or longer, red indicator will appear and the device enter an X-ray locked state.

Button	Description
	<p>[X-ray Exposure]</p> <ul style="list-style-type: none"> ▪ Used for X-ray exposure. ▪ Press the button until the beep sound stops and it will exposure 1 time as per set exposure time. <hr/> <p> Releasing the button during exposure immediately stops X-ray exposure and movement of the instrument.</p>
	<p>[Reset]</p> <ul style="list-style-type: none"> ▪ Used for forced shut down and updating the equipment.
	<p>The buttons are basically used as above, but can be used differently for each menu.</p> <p>Menu-specific button functions are shown at the bottom of the "Display."</p> <p>The underlined function in the display is activated when the button is pressed for at least 2 seconds.</p>

2.3 Exposure Restriction Mechanism

- Exposure Restriction Mechanism

Image	Description
	<p>This device is equipped to prevent excessive exposure of the X-ray.</p> <p>The inside of the X-ray tube adopted a lead shielding and an aluminum filter. The cone is used to minimize the X-ray exposure to the operator and the patient by adjusting the distance between the equipment and the patient.</p>

2.4 Connector

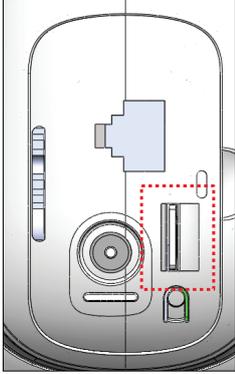
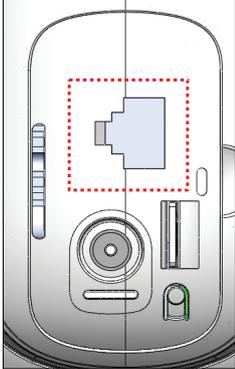
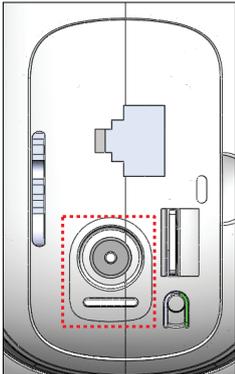
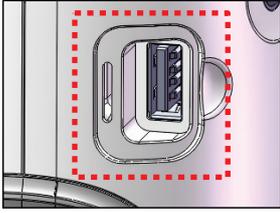
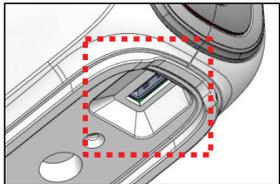
Image	Description
	<p>[Micro SD Card connector]</p> <ul style="list-style-type: none"> A connector that inserts the Micro SD Card. <hr/> <p> CAUTION It is encouraged to have important clips be backed up elsewhere other than the Micro SD card, as our company does not have any responsibility for any form of loss or damage of recorded data on the Micro SD card.</p> <hr/> <p> CAUTION Our company does not provide warranty for any errors with Micro SD cards other than the ones we provide.</p>
	<p>[Hand switch connector & mA measurement connector]</p> <ul style="list-style-type: none"> A connector that connects the hand switch. It is usable when measuring mA from the outside. <hr/> <p> CAUTION You can only use the hand switch connector provided by the company.</p>
	<p>[Charger connector]</p> <ul style="list-style-type: none"> A connector that is used to charge the battery. <ul style="list-style-type: none"> If you are charging from 0% battery, allow up to 2 hours and 30 minutes to fully charge. <hr/> <p> The charging time may vary based on the battery percentage.</p> <hr/> <p> CAUTION You can only use the charging adaptor provided by the company.</p> <hr/> <p> CAUTION Do not charge the device for too long, exceeding the recommended charging time.</p>

Image	Description
	<p>[Oral sensor USB connector]</p> <ul style="list-style-type: none"> ▪ A USB connector that connects the oral sensor. <hr/> <p> You can only use the oral sensor provided by our company.</p> <p>CAUTION Furthermore, the oral sensor needs to be permitted with 5 V.d.c provided by our company.</p>
	<p>[Holder string]</p> <ul style="list-style-type: none"> ▪ In order to use the equipment securely, hold the equipment firmly with your hand. ▪ The string length can be adjusted for the user's convenience.
	<p>[Wireless USB dongle connector]</p> <ul style="list-style-type: none"> ▪ A USB connector to insert a wireless USB Dongle. <hr/> <p> The company encourages the usage of a wireless USB dongle provided by us.</p> <p>CAUTION The company does not provide warranty for any errors caused from using a wireless USB dongle that is not provided by us.</p> <hr/> <p> The wireless environment and network status determines the speed of the transfer of data.</p> <p>CAUTION</p> <hr/> <p> The device will not function properly if there is a jammer.</p> <p>CAUTION</p>

2.5 Battery

- This device adopts a lithium-polymer battery, which requires regular maintenance and management for its use and handling.
- Repeated charging shortens the available time for use.

As an approximate standard, after the battery has been charged about 500 times, the possible time it can be used is around 70-80% of what was possible at the time it was first purchased.

- For a long-term storage, recharging of the battery once for every six months is strongly recommended.

Due to the nature of lithium-polymer batteries, the remaining battery capacity reduces even when not in use.

- Please note that if the equipment is not used for a long time, there may be cases where charging does not take place even when the charger is connected.
- The charging time and usage time may differ depending on the usage environment.
- If the remaining battery power is insufficient, the amount of X-ray radiation may be insufficient during shooting, so you should always maintain the remaining power.
- When the battery reaches the lowest limit due to insufficient battery power, the remaining battery indicator flashes on the LCD screen, and after a certain period of time, the power of the device is automatically turned off.
- Keeping the battery fully charged, totally empty, or using the device until it runs out of power drastically diminishes the life of the battery.
- Exposure to a very hot environment also drastically diminishes the life of the battery.



2 hours is enough for an emptied battery to be fully recharged.



While charging the battery, X-ray exposure is disabled.
The message "Warning 00" displayed.

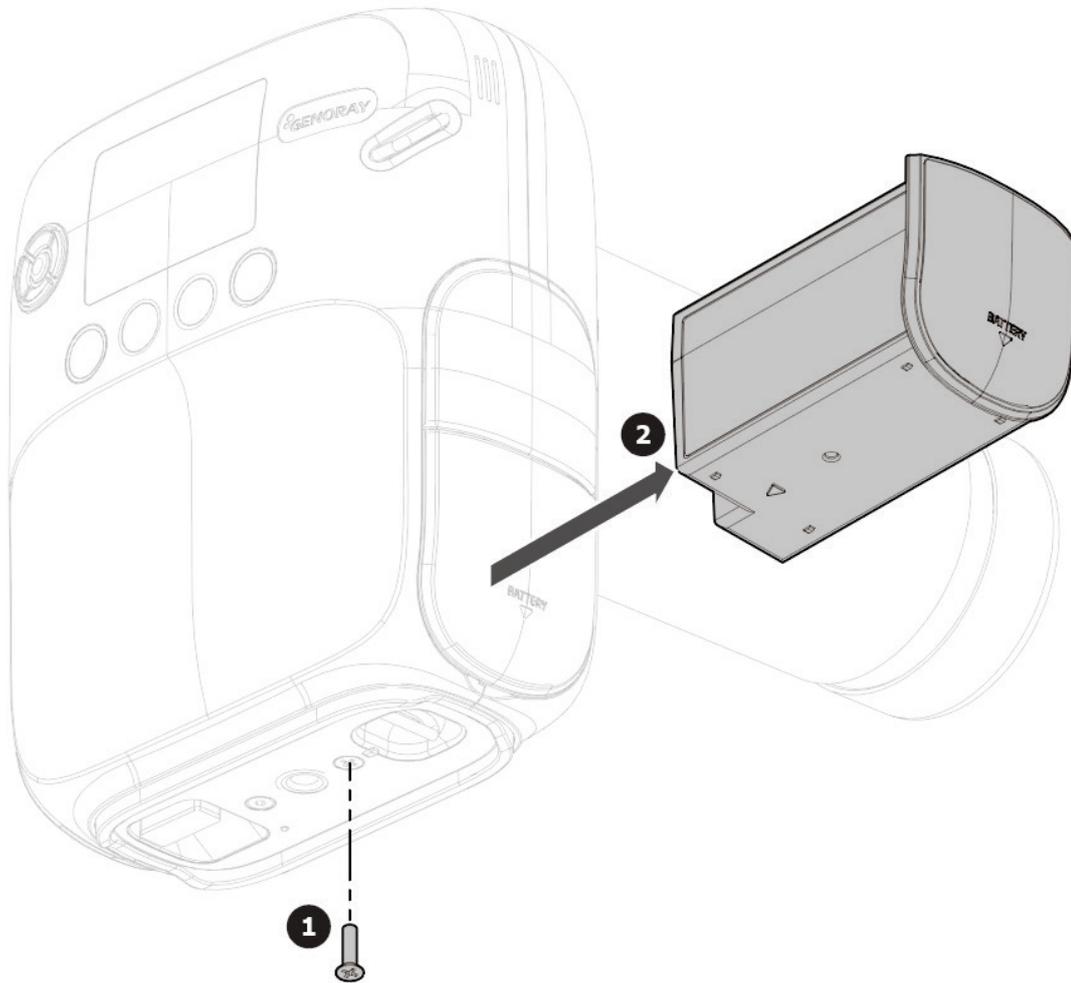


Use the battery supplied by GENORAY only.
The operator is responsible for any malfunction from use with non-approved battery.



Battery maintenance may vary depending on the device.

How to replace the battery



1. Use a (+) screwdriver to remove the battery fixing bolts (1 pcs) from the bottom of the equipment.
2. Remove the battery.
3. Replace the battery and assemble the battery in the reverse order of disassembly.



A battery that is too swollen to be detached from the device can be dangerous. Do not attempt to puncture, bend, or crush the battery to forcibly detach it. Contact the service center.



If the fluid inside the battery leaks and makes contact with the skin or an eye, immediately rinse the affected area thoroughly with running water and call the doctor.



- Please dispose used battery as general waste.
- Dispose of used batteries according to the laws and regulations of each country or region.

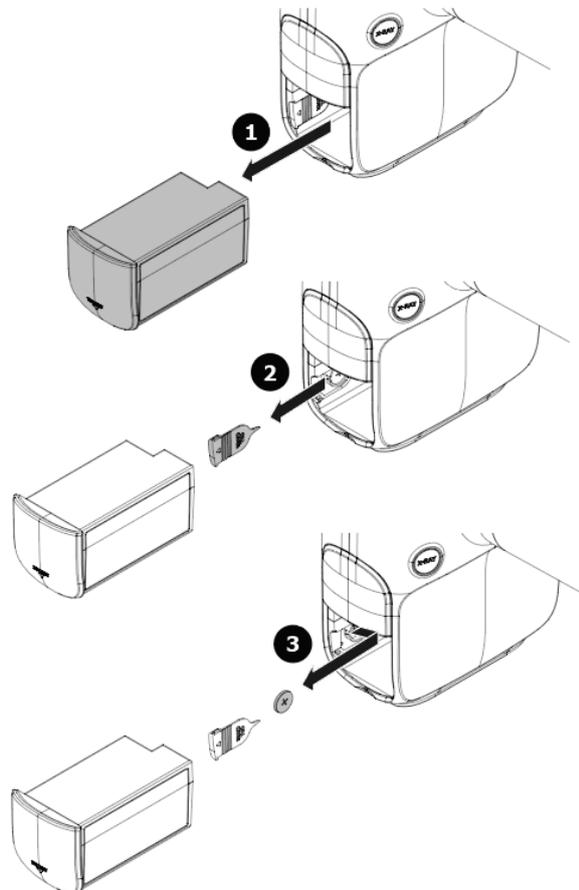
2.5.1 Coin cell

- The Coin cell inside the equipment maintains the time of the system while the equipment is turned off.



The life of the coin cell can vary depending on the manufacturer or the date of manufacture.

Replacement of the Coin Cell



1. Detach the battery of the equipment.
2. Detach the Coin cell cover inside the equipment.
3. Detach the old coin cell using the cell cover and mount a new one.

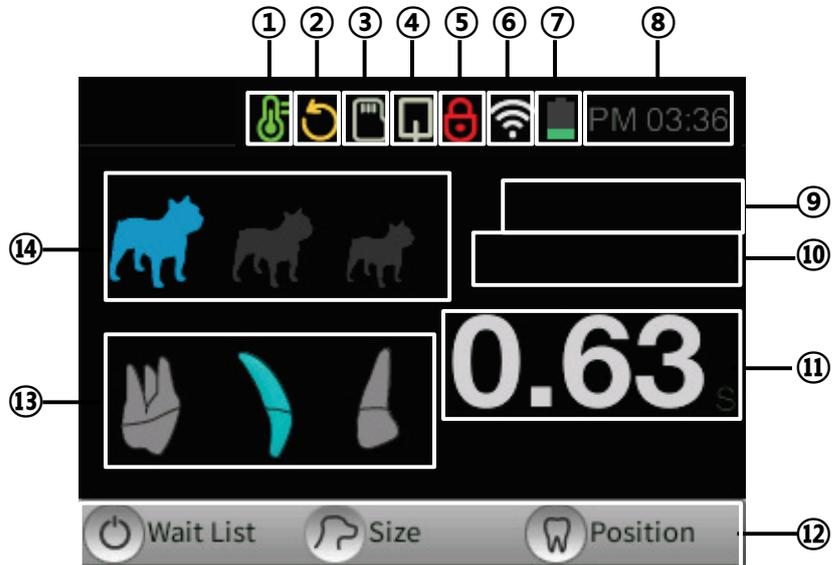
In doing so, the anode of the cell should face up.



- The specification of the button cell is imprinted on the surface of the cell.
- Make sure to replace the cell with the new one of the same specification.
 - Coin cell specification: CR1220

2.6 Display

You can check the device status and filming condition on the display.



No.	Feature	No.	Feature
1	X-ray tube temperature check	8	Current time
2	Firmware update notification	9	Patient name
3	Micro SD card insert check	10	Patient number, Patient sex
4	Oral sensor connection check	11	X-ray exposure time
5	X-ray exposure lock notification	12	Button feature description
6	Wireless (Wi-Fi) connection check	13	Teeth placement selection
*7	Battery check	14	Patient size selection

***7 Battery Check**

Equipment in use		Charging	
	Low battery level		Battery charging (10%)
	Battery 50% left		Battery charging (50%)
	Battery 100% left		Battery fully charged



It is recommended to maintain the remaining battery level at 50% or more for smooth use of the equipment.

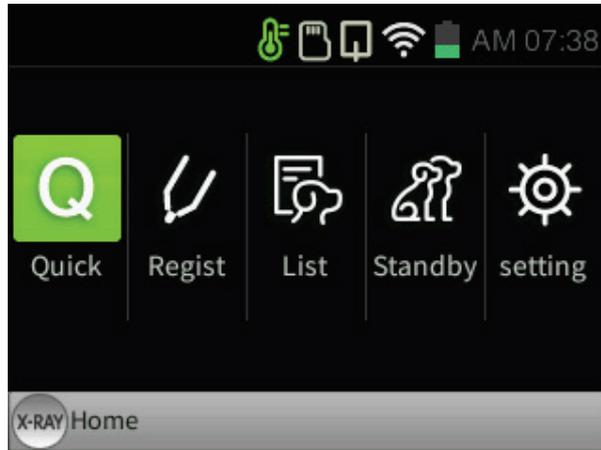


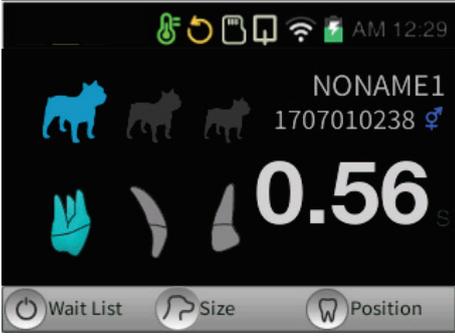
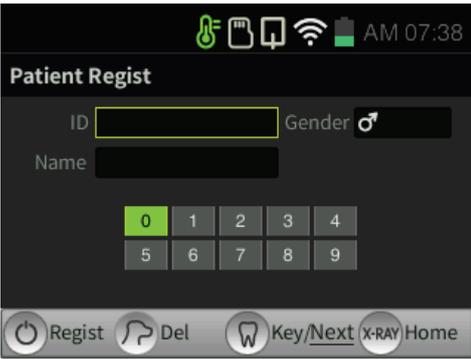
When the main system is connected to the charger, an alarm occurs on the bar above the display for a second, as below:

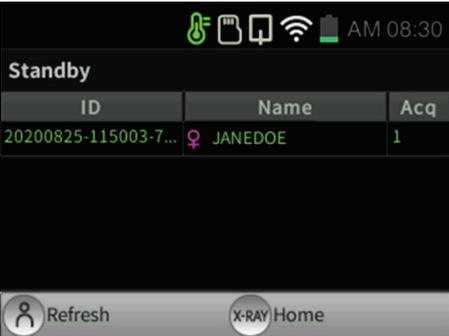
- When connected: remaining time to be fully charged.
- When disconnected: available use time (standby time)

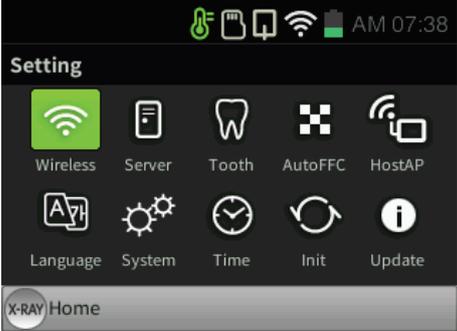
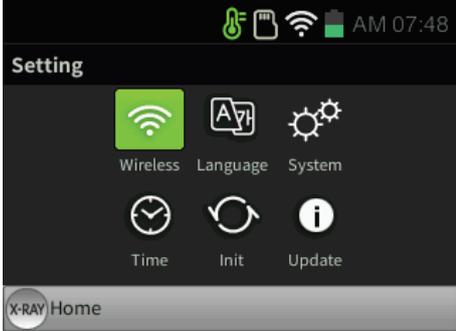
2.6.1 Menu

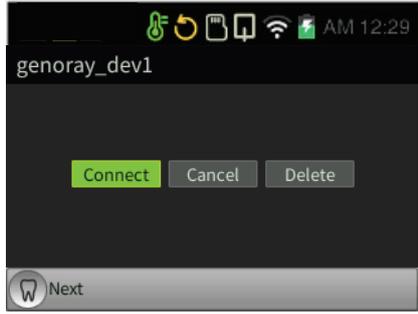
1. When the menu button () is pressed, the following screen will be displayed.
2. If you want to return to the main screen from the menu screen, click the X-ray exposure button.

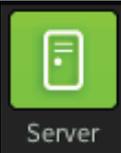
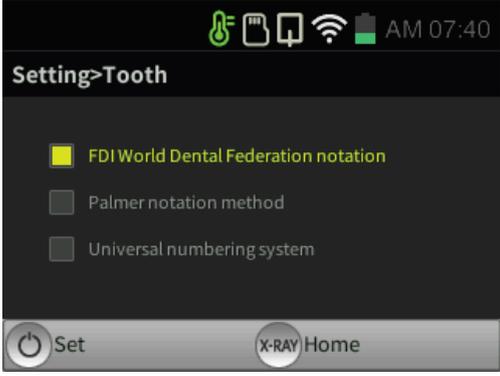


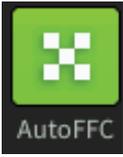
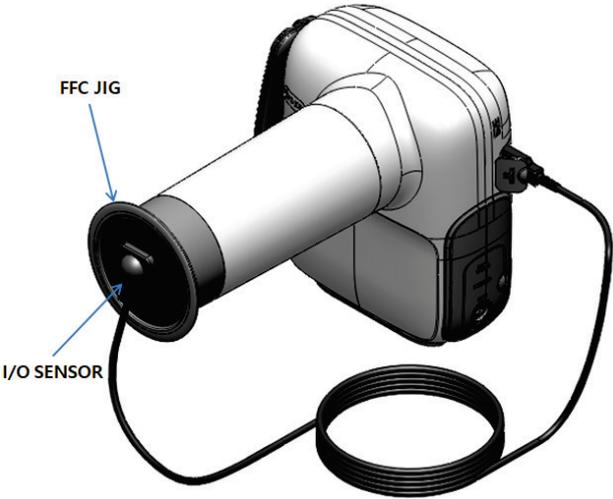
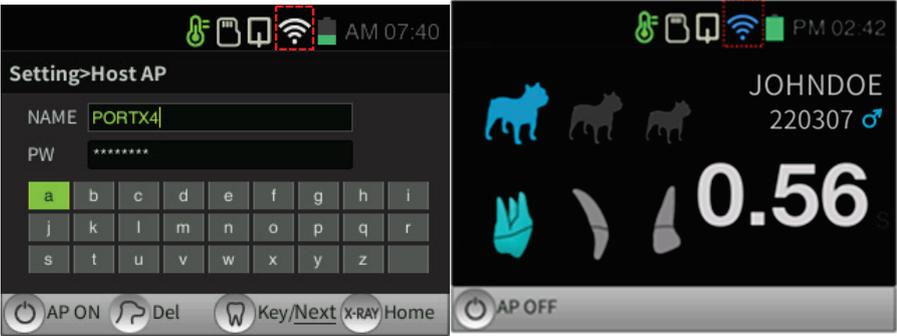
Menu	Description
	<p>[Quick]</p> <ul style="list-style-type: none"> ▪ Click on Quick button to automatically register patients. ▪ Patient number: It is registered as "Registration Time", "Patient Name: NONAME", "Sex: Neutral." 
	<p>[Regist]</p> <ul style="list-style-type: none"> ▪ The patient registration window will pop up when you click Regist button. ▪ After inputting patient information, click the power button () to register. 

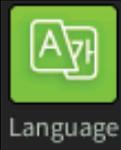
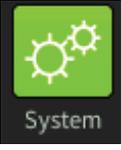
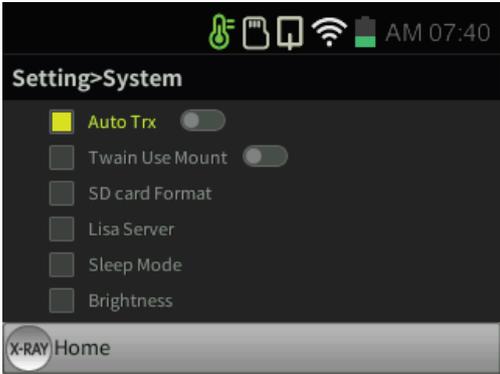
Menu	Description
	<p>[List]</p> <ul style="list-style-type: none"> The patient list will pop up when you click on List button. If you click patient information using the the trackball() or select button () , an image thumbnail view will pop up.  <hr/> <ul style="list-style-type: none">  The "Send" feature requires wireless settings and a server PC to be set.  You can register up to 200 patients and save 100 images per patient.  You can save up to approximately 3400 films with a 16 GB Micro SD card.
	<p>[Standby]</p> <ul style="list-style-type: none"> When you click Standby button, the list of patients who requested for image will pop up on PORTVIEW.  <hr/> <ul style="list-style-type: none">  The "Standby" feature <ul style="list-style-type: none"> Can only be linked with a PORTVIEW version higher than 2.2.5.0. Wireless network settings and a server PC need to be set.  If multiple devices are using a server PC at the same time and you are trying to select patients, you will not be able to select multiple patients.

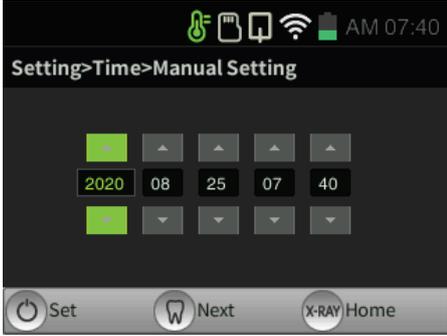
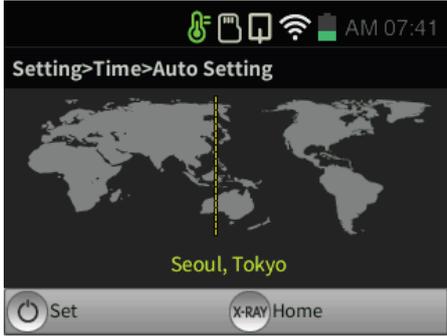
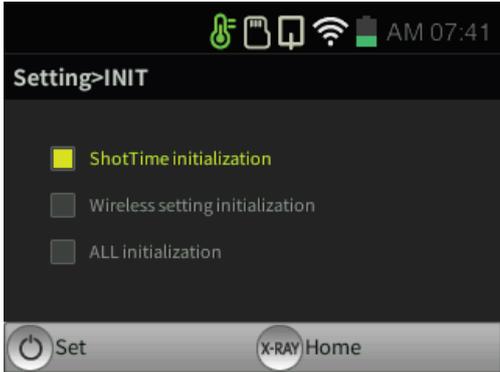
Menu	Description
	<p>[Setting]</p> <ul style="list-style-type: none"> ▪ The settings window will pop up when you click on Setting. ▪ The settings window is divided into “when using the oral sensor” and “when not using the oral sensor.” You can set the usage environment of PORT-X IV. <div style="display: flex; justify-content: space-around;"> <div style="text-align: center;">  <p>[using the oral sensor]</p> </div> <div style="text-align: center;">  <p>[not using the oral sensor]</p> </div> </div>

Menu	Description
	<p>[Wireless]</p> <ul style="list-style-type: none"> The wireless connection method is as follows. <hr/> <div style="display: flex; justify-content: space-between;"> <div style="width: 48%;"> <p>① Settings -> Select Wireless</p>  </div> <div style="width: 48%;"> <p>② Check the wireless AP list and select the wireless AP to use. When linking with PortView, connect to the same wireless AP with the server PC</p>  </div> </div> <hr/> <div style="display: flex; justify-content: space-between;"> <div style="width: 48%;"> <p>③ Enter the password for the selected wireless AP.</p>  </div> <div style="width: 48%;"> <p>④ Connect to the wireless connection.</p>  </div> </div> <hr/> <div style="margin-top: 20px;"> <p> If the connection is not stable, you can set the settings to manual if you click on the register patient button.</p> <hr/> <p> You can only use networks with passwords.</p> </div>

Menu	Description
	<p>[Server]</p> <ul style="list-style-type: none"> ▪ You can select Server PC to connect to a server PC. <ul style="list-style-type: none"> • Basic ID: PORTXIV PW: 12345678 ▪ The IP address has to be the same as the server PC. <div data-bbox="671 427 1171 797" style="border: 1px solid black; padding: 5px; margin: 10px 0;">  </div> <hr/> <div data-bbox="517 846 612 943" style="display: inline-block; vertical-align: middle;">  </div> <div data-bbox="651 846 1337 943" style="display: inline-block; vertical-align: middle;"> <p>If multiple devices are using a server PC at the same time and you are trying to select patients, you will not be able to select multiple patients.</p> </div> <hr/> <div data-bbox="517 994 612 1090" style="display: inline-block; vertical-align: middle;">  </div> <div data-bbox="651 994 1347 1090" style="display: inline-block; vertical-align: middle;"> <p>See Appendix for the server PC ID, PW change, and server IP address.</p> </div>
	<p>[Tooth]</p> <ul style="list-style-type: none"> ▪ You can set the teeth numbers. <div data-bbox="671 1196 1171 1572" style="border: 1px solid black; padding: 5px; margin: 10px 0;">  </div>

Menu	Description
	<p>[Auto FFC]</p> <ul style="list-style-type: none"> If the message that there is no FFC file is displayed when reconnecting, click the Create FFC button again to create it.  <hr/>  Please note that x-rays are automatically generated when creating a FFC.
	<p>[HostAP]</p> <ul style="list-style-type: none"> You can connect the mobile app and equipment to each other. <ul style="list-style-type: none"> Basic Name: PORTXIV Basic PW: 12345678 The wireless icon changes color when it is properly connected.  <p style="text-align: center;">[not connected] [connected]</p> <hr/>  Supported devices: Android, Android tablet, iPhone, iPad.

Menu	Description
	<p>[Language]</p> <ul style="list-style-type: none"> You can set the language of the equipment. Settable languages: Korean, English, Japanese
	<p>[System]</p> <ul style="list-style-type: none"> Automatic transfer When you check automatic transfer, the data is transferred automatically and sent to the server. Twain use Mount Exposure without tooth number when using Twain Format SD card When you check Format SD card, all of the patient information and films will be deleted. Lisa Server If you check Lisa Server, all the patient information is sent to the Lisa Server. Power saving mode Set the power saving time and equipment OFF time. Brightness adjustment Adjust the brightness of the LCD. <div data-bbox="671 1077 1171 1451" style="border: 1px solid black; padding: 5px; margin: 10px 0;">  </div> <hr/> <div data-bbox="517 1503 612 1599" style="display: inline-block; vertical-align: middle;">  </div> <div data-bbox="651 1518 1347 1585" style="display: inline-block; vertical-align: middle; margin-left: 10px;"> <p>When formatting the Micro SD card, all patient information and films are deleted.</p> </div>

Menu	Description
	<p>[Time]</p> <ul style="list-style-type: none"> ▪ You can set the time. <ul style="list-style-type: none"> • Manual setting <p>Select Settings, and control the time using the the trackball() or move button ()</p>  • Automatic setting <p>If you select a country when connected to the Internet, the time is automatically set to the time of the selected country.</p> 
	<p>[Init]</p> <ul style="list-style-type: none"> ▪ Film time reset <p>Initialize the X-ray imaging time.</p> ▪ Wi-Fi reset <p>The Wi-Fi network settings are reset.</p> ▪ Overall reset <p>All of the settings are reset.</p> 

Menu	Description
	<p>[Update]</p> <ul style="list-style-type: none"> ▪ You can check current version of the firmware and also update it to the latest version. ▪ Check firmware version You can check the version of the firmware by clicking on the teeth select button. ▪ How to Update Firmware <ol style="list-style-type: none"> ① Check the version. ② Click the power button (⏻) to proceed updating. ③ The device will be automatically turned off when the update is completed. ④ Click the power button (⏻) to restart the device. <hr/> <p>The update environment is as follows.</p> <ul style="list-style-type: none"> ▪ Wireless network connection ▪ Insert the Micro SD card ▪ Micro SD capacity: at least 50 Mb ▪ Battery must be higher than 50% or has to be charging. <hr/> <p> CAUTION Do not use any other firmware updates that are not provided by the manufacturer. It can damage the device. Any damage caused by using firmware updates that are not provided by the manufacturer are not covered by warranty.</p> <hr/> <p> If you disconnect the battery or use an unstable charging port, the device may be damaged.</p>

3. Directions

3.1 Preparation before use

- Get well-acquainted with the user manual.
- Check the voltage specified by the manufacturer.
- Check the charging level before use.



X-ray exposure is not possible while the device is charging.



Before imaging, you must ask the patient to remove other jewelry and metallic items worn by them.

Performing imaging which such objects are present may cause shadows and reflections in the image.



The operator should do the imaging from a location safely protected from X-rays and while wearing protective equipment.



Be careful not to release the exposure button during imaging.

Releasing the exposure button will stop X-ray exposure and you will receive a message about an abnormal shutdown of the equipment.



Pay attention to the condition of the patient and the equipment during the imaging.

In the case of an unexpected problem, immediately release the exposure button.

3.2 Image Acquisition

3.2.1 Filming Method - Not Using the Sensor

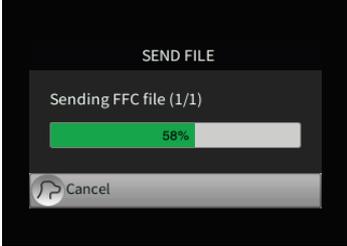
Sequence	Description	Notes
<p>1. Setting X-ray exposure time</p>	<p>① Click the patient size button to set the size of the patient. Patient size button ()</p> <p>② After clicking the tooth selection button, set the location for imaging. Tooth selection button ()</p> <p>③ When selecting patient size and tooth, the pre-stored X-ray exposure time is set</p> <hr/> <p>If, after the end of X-ray exposure, the image is not satisfactory due to an insufficient or excessive X-ray dose, You can adjust the X-ray exposure time by moving the trackball() or move button () up or down.</p> <ul style="list-style-type: none"> ▪ ↑ : Increase X-ray exposure time (Large/Medium/Small: 30 Step) ▪ ↓ : Reduce X-ray exposure time (Large/Medium/Small: 30 Step) <hr/> <p> After changing the exposure time set as above, the changed X-ray exposure time is saved simultaneously at the end of exposure.</p>	  <div style="background-color: black; color: white; padding: 10px; font-size: 24px; font-weight: bold; text-align: center;">0.63</div>
<p>2. Equipment Alignment</p>	<p>① Accurately place the exposure restriction equipment at the site to be photographed</p>	

Sequence	Description	Notes
3. Imaging	① Press the X-ray exposure button.	
	② A "Ready" message appears in the LCD window for about 2 seconds.	
	③ When the exposure begins, an "X-ray exposure" notification window is displayed on the LCD, and a beep sound occurs.	
	 You have to press the X-ray exposure button for the set X-ray exposure time.	
	④ When the exposure is completed, the beep sound ends and the X-ray exposure notification window disappears.	
	 <ul style="list-style-type: none"> ▪ Duty Cycle of Exposure Interval time <ul style="list-style-type: none"> • X-ray exposure time is limited for about 30 times the exposure time. • Ex) Maximum exposure time (1.6 seconds) X-ray exposure is limited for approximately 50 seconds. 	

3.2.2 Filming Method - Using the Sensor

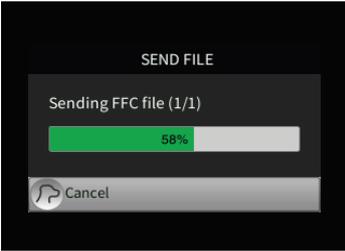
PORT-X IV Filming Technique

Sequence	Description	Notes
<p>1. Register patients</p>	<p>① Register or select the patient getting imaged. Menu → "Quick", "Regist", "List"</p>	<p>Refer to "2.6.1"</p>
<p>2. Set X-ray exposure time</p>	<p>① Set the size of the patient by clicking on the patient size button</p> <p>Patient size button ()</p> <p>② After clicking the tooth selection button, set the location for imaging.</p> <p>Tooth selection button ()</p> <p>③ When selecting patient size and teeth, the previously saved x-ray exposure time will be set.</p> <hr/> <p>If you are not satisfied with the image due to lack of x-rays or excessive amount of x-rays after completing the x-ray exposure, you can control the exposure time with the up/down movement of the trackball() or move button ().</p> <ul style="list-style-type: none"> ▪ ↑ : Increase X-ray exposure time (Large/Medium/Small: 30 Step) ▪ ↓ : Reduce X-ray exposure time (Large/Medium/Small: 30 Step) <hr/> <p> After changing the exposure time as above and taking the X-ray, the modified exposure time will be saved simultaneously.</p>	<div style="display: flex; flex-direction: column; align-items: center;">   <div style="border: 2px solid black; padding: 10px; margin: 20px 0;">0.63</div> </div>
<p>3. Device alignment</p>	<p>① Place the device exactly where you wish to locate.</p>	

Sequence	Description	Notes
<p>4. Take the image</p>	<p>① Click on the x-ray exposure button</p> <p>② On the LCD screen, "Ready" pop-up window will appear for about 2 seconds.</p> <p>③ Once the exposure starts, the "X-ray exposure" pop-up will show up on the LCD screen and the beeping will begin.</p> <hr/> <p> You are required to press on the X-ray button for the amount of time set to perform the X-ray.</p> <hr/> <p>④ Once the exposure is over, the beeping will stop and X-ray exposure pop up screen will disappear.</p> <hr/> <ul style="list-style-type: none"> ▪ Duty Cycle of Exposure Interval time <ul style="list-style-type: none"> • X-ray exposure time is limited for about 30 times the exposure time. • Ex) Maximum exposure time (1.6 seconds) X-ray exposure is limited for approximately 50 seconds. <p></p>	  
<p>5. Check the image</p>	<p>① Check the image once imaging is completed.</p> <ul style="list-style-type: none"> • The captured image is automatically saved to the Micro SD card. • When automatic transmission is selected, the image is automatically sent to the server. <div style="display: flex; justify-content: space-around;">   </div>	

Server Linked Imaging Method

Sequence	Description	Notes
<p>1. Select Standby Patients</p>	<p>① Select the standby patient that has been registered on the server.</p> <ul style="list-style-type: none"> • Menu → "Standby" • Click the power button on the main screen. 	<p>Refer to 2.6.1</p>
<p>2. X-ray exposure time setting</p>	<p>① The size of the patient and placement of teeth are automatically set to the site designated by PC.</p> <p>② When selecting patient size, the previously saved X-ray exposure time will be set.</p> <hr/> <p>If you are not satisfied with the image due to lack of x-rays or excessive amount of x-rays after completing the x-ray exposure, you can control the exposure time with the up/down movement of the trackball() or move button ().</p> <ul style="list-style-type: none"> ▪ ↑ : Increase X-ray exposure time (Large/Medium/Small: 30 Step) ▪ ↓ : Reduce X-ray exposure time (Large/Medium/Small: 30 Step) <hr/> <p> After changing the exposure time as above and taking the X-ray, the modified exposure time will be saved simultaneously.</p>	<div style="background-color: black; color: white; padding: 10px; text-align: center; font-size: 2em; font-weight: bold;">0.63</div>
<p>3. Device alignment</p>	<p>① Place the device exactly where you are going to image.</p>	

Sequence	Description	Notes
<p>4. Take the image</p>	<p>① Click on the x-ray exposure button</p> <p>② On the LCD screen, "Ready" pop-up window will appear for about 2 seconds.</p> <p>③ Once the exposure starts, the "X-ray exposure" pop-up will show up on the LCD screen and the beeping will begin.</p> <hr/> <p> You are required to press on the X-ray button for the amount of time set to perform the X-ray.</p> <hr/> <p>④ Once the exposure is over, the beeping will stop and X-ray exposure pop up screen will disappear.</p> <hr/> <p> <ul style="list-style-type: none"> ▪ Duty Cycle of Exposure Interval time <ul style="list-style-type: none"> • X-ray exposure time is limited for about 30 times the exposure time. • Ex) Maximum exposure time (1.6 seconds) X-ray exposure is limited for approximately 50 seconds. </p>	<p></p> <p></p> <p></p>
<p>5. Check the image and send</p>	<p>① Check the image once imaging is completed. The captured image is automatically saved to the Micro SD card.</p> <p>② Send the file to the server.</p> <p>Click on the Select Teeth button () to begin transfer.</p> <div style="display: flex; justify-content: space-around;">   </div>	

3.3 Image Transfer

Send images from PORT-X IV to the server PC.



Based on the distance of the router and connection stability, the transfer time may differ.

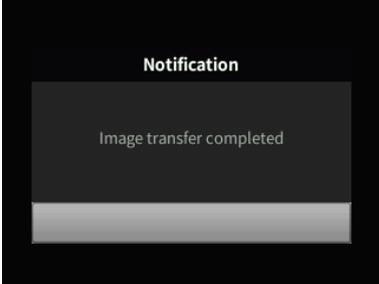


When you are transferring the file for the first time, the FFC file will be transferred, which may require additional time.

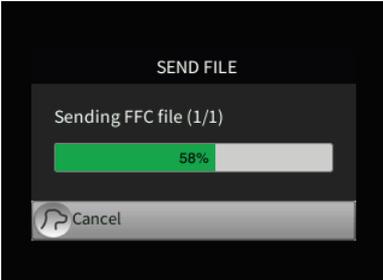
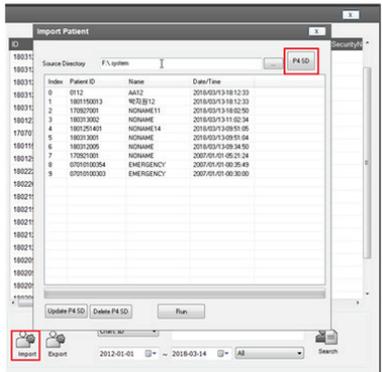


There are two types of transfers: automatic and manual. These settings can be changed in Menu→System. (Refer to 2.6.1)

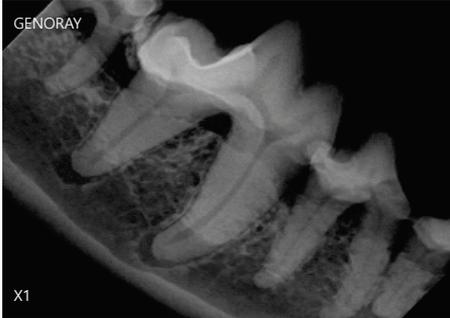
3.3.1 Automatic Transfer

Logistics	Description	Notes
<p>1. Complete imaging</p>	<p>① After exposure, the image and the progress bar will appear.</p>	
<p>2. Complete transfer</p>	<p>② Once the transfer is completed, move on to the next image.</p> <ul style="list-style-type: none"> • For multiple images for one patient, go to the main screen. • Once imaging is completed, go to Standby Patient List. • For re-imaging, go to PortView and request imaging. 	

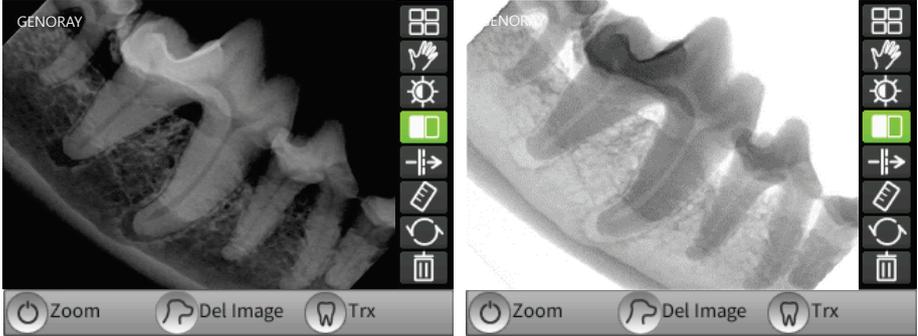
3.3.2 Manual Transfer

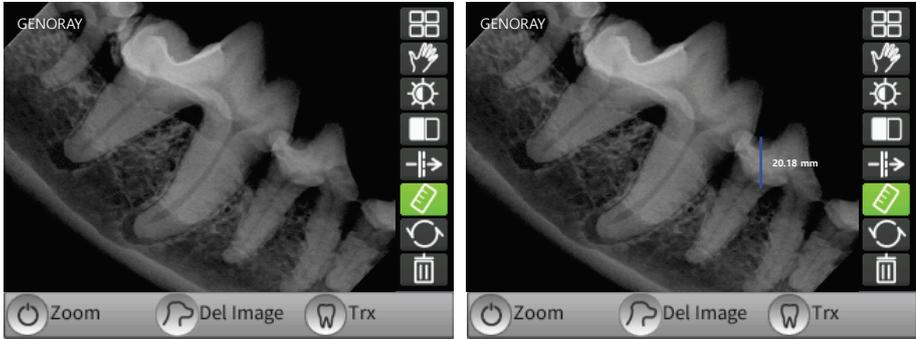
Logistics	Description	Notes
<p>1. Complete imaging</p>	<p>① Once imaging is completed, the captured image will be printed. For re-imaging, press X-ray exposure button to conduct re-imaging.</p>	
<p>2. Complete transfer</p>	<p>① Transfer the image to the SD card by tapping the patient size button. (👉)</p> <p>② Detach the SD card from the equipment and attach it to the SD card reader on the PC.</p> <p>③ Launch PortView and go to the Patient Management window.</p> <p>④ From the bottom of the Patient Management window, click the [Import] button to input the image.</p> <p>⑤ Enter the path to the SD card and click the [P4 SD] button to show the patient list.</p> <ul style="list-style-type: none"> Select the patient and click the [Run] button to finish transferring the patient's image. 	 

3.4 Image Management

Menu	Description
Image change	<p>[Change]</p> <ul style="list-style-type: none"> The image can be changed by clicking the patient size button (👤). Briefly pushing the button changes the image to the next one. Holding the button changes the image back to the previous one.
Transfer image	<p>[Transfer]</p> <ul style="list-style-type: none"> Click on Select Imaging Location button (📍) to transfer the images to the server PC. 
Zoom-in the image	<p>[Zoom-in]</p> <ul style="list-style-type: none"> Click the power button (🔌) to magnify the image in a total of 3 steps (1x, 1.3x, 1.7x). <div style="display: flex; justify-content: space-around;"> <div data-bbox="453 1245 903 1563">  <p>X1</p> </div> <div data-bbox="916 1245 1366 1563">  <p>X1.69</p> </div> </div> <div style="display: flex; justify-content: space-around; margin-top: 10px;"> <p>[before zoom]</p> <p>[after zoom]</p> </div> <hr/> <div style="display: flex; align-items: center; margin-top: 10px;">  <p>If you press Zoom-in button again at 1.7x, it returns to the 1x image.</p> </div>

Menu	Description
Image list	<p>[List]</p> <ol style="list-style-type: none"> Click the menu button () to bring up the image management tab. Click the list button to go to the thumbnail screen. 
Moving image	<p>[Moving]</p> <ol style="list-style-type: none"> Click the menu button () to bring up the image management tab. After clicking the move button, you can move the image using the trackball() or move button ()  <hr/>  Moving the image is only available when the image is magnified.

Menu	Description
<p>Brightness adjustment</p>	<p>[Brightness Adjustment]</p> <ol style="list-style-type: none"> Click the menu button (☰/⊗) to bring up the image management tab. After clicking the brightness control button, you can adjust the brightness using the trackball (⦿) or move button (⦿).  <hr/>  <p>The modified screen brightness is automatically saved.</p>
<p>Color inversion</p>	<p>[Color Inversion]</p> <ol style="list-style-type: none"> Press the menu button (☰/⊗) to bring up the image management tab. Click the color inversion button to invert the color of the image.  <p>[Pre- inversion] [Post- inversion]</p>

Menu	Description
Filter	<p>[Filter]</p> <ol style="list-style-type: none"> Click the menu button (☰) to bring up the image management tab. By pressing the filter button, you can select the filter level from Level 0-3. 
Length measurement	<p>[Length Measurement]</p> <ol style="list-style-type: none"> Click the menu button (☰) to bring up the image management tab. When you click the length measurement button, a red dot is displayed on the image. You can move the dot using the trackball (⦿) or move button (⦿). When you click the trackball (⦿) or select button (⦿) at a desired spot, the measured length is displayed.  <hr/> <p> The final length, which is the sum of the dot's moving lengths, is output.</p> <hr/> <p> Dots can be marked multiple times, and the most recently selected dot can be deleted by clicking the Patient Size button (👤).</p>

Menu	Description
Initialization	<p>[Initialization]</p> <ol style="list-style-type: none"> Click the menu button (☰) to bring up the image management tab. You can reset the modified image to the original image by clicking the reset button.  <hr/>  <p>Zoom, movement, and brightness will be all reset to the original image.</p>
Delete image	<p>[Delete Image]</p> <ol style="list-style-type: none"> Click the menu button (☰) to bring up the image management tab. Click the patient size button (👤) to delete the image. 

4. Precautions after Use

4.1 Regular Inspection

- It is recommended that this equipment is inspected by the service center at least once a year.
- This equipment is subject to equipment inspection and defense inspections once every three years by the inspection agency of the Ministry of Food and Drug Safety in accordance with the regulations on radiation safety control.

4.2 Care and Cleaning

- The performance of the device and its components must be checked regularly. See "Appendix 4. Maintenance."
- If the equipment has not been used for a long time, inspect the unit to make sure that it is operating normally. Please call the service center for inspection.
- If the equipment requires cleaning, turn off the power and then use a soft cloth or gauze to wipe it without using force avoid damages to the equipment.

This page is intentionally left blank.

5. Equipment Specifications

5.1 System

Model name

No.	Item	Specification
1	Basic model name	PORT-X IV
2	Multiple model name	ZEN-PX4

Technical data

No.	Item	Specification
1	Rated power	100-240 V~
2	Power consumption	180 W
3	Frequency	50/60 Hz

Environment

No.	Item		Specification
1	In Use	Temperatures	+6 to +35 °C
		Humidity	10 % to 80 %RH(no dew condensation)
		Atmospheric pressure	800 to 1060 hPa
		Temperatures	+6 to +35 °C
2	In Storage	Temperatures	-10 to 60 °C
		Humidity	10 % to 80 % RH(no dew condensation)
		Atmospheric pressure	500 to 1060 hPa

5.2 X-ray Generator

Input

No.	Item	Specification
1	Type	HF Inverter 38.5 kHz
2	Phase	Single

Output

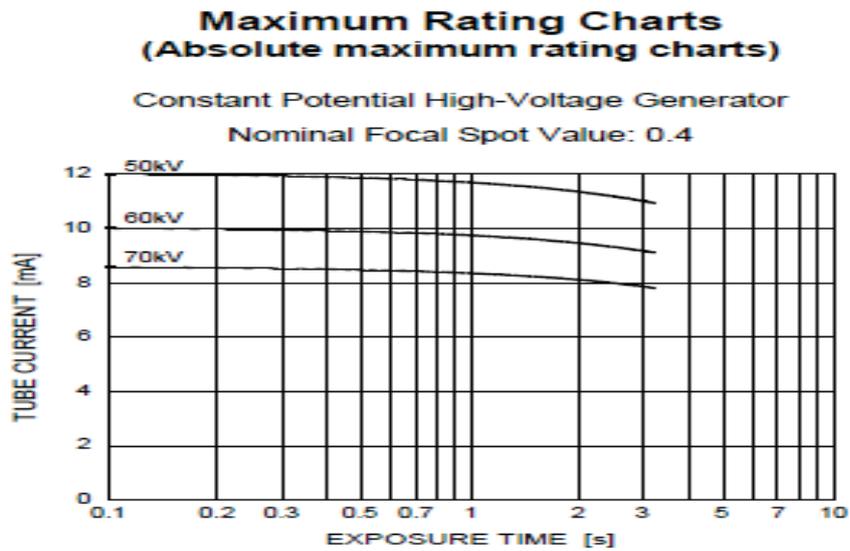
No.	Item	Specification
1	Nominal Peak Output Power	140 VA
2	kV range	70 kV(Fixed)
3	mA range	2 mA(Fixed)
4	Scan/Exposure time(sec)	0.05-1.6

5.3 X-ray Tube

D-045S

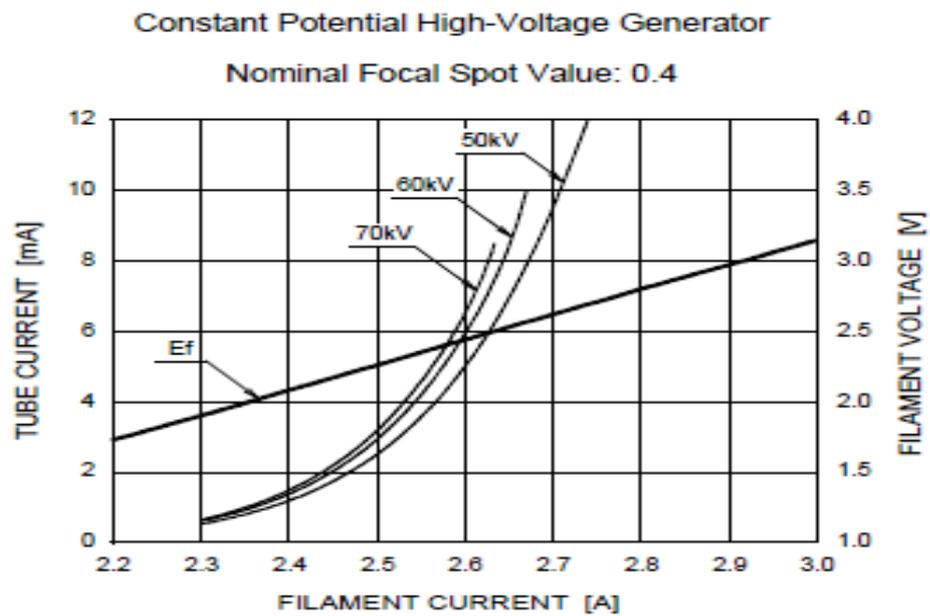
No.	Item		Specification
1	Electrical Data	Circuit	DC(Center Grounded)
		Focus	0.4 mm
2	Mechanical Data	Tube type	Stationary
		Target angle	12.5 °
		Duty cycle	1:30 or more
		Total filtration	-
		Inherent filtration	1.0 mm Al
3	Max. Assessment	Max. Tube voltage	70 kV
		Anode Heat Capacity :	4.3 kJ
		Tube Housing Heat Storage Capacity	18.06 KHU (12.9 kJ)

Maximum Rating Charts (Absolute Maximum Rating Charts)



Emission Characteristics

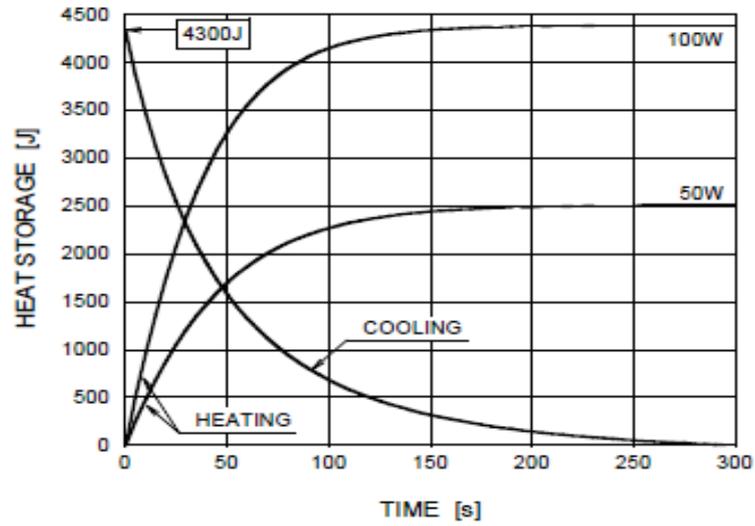
Emission & Filament Characteristics



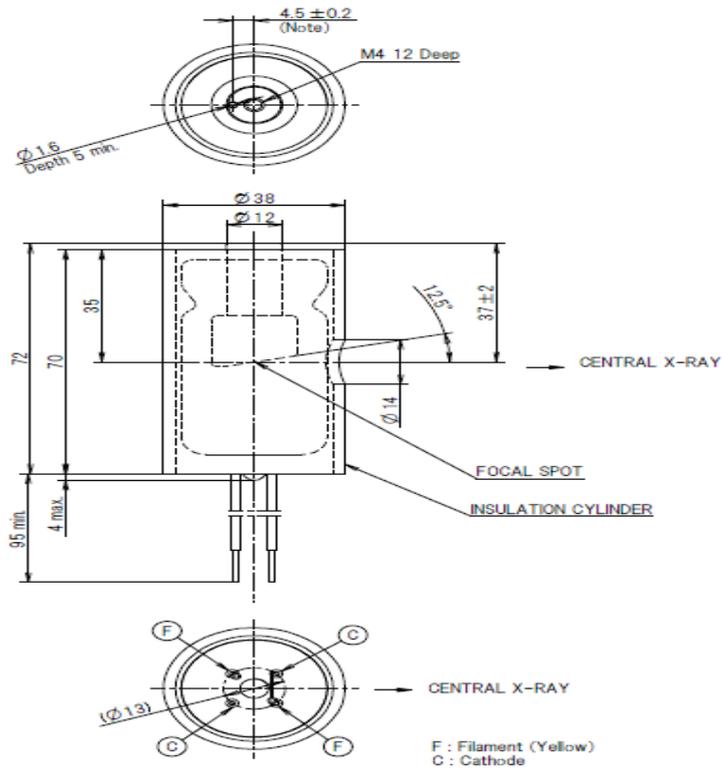
Note: This graph indicates typical characteristics.

Anode Thermal Characteristics

Anode Heating / Cooling Curve



D-045S Dimensions (Unit: mm)



Note : Dimensions from an anode shank to a mounting hole.

5.4 Battery

No.	Item	Specification
1	Type	Li-polyper
2	Voltage	22.2 VDC
3	Capacity	1,000 mAh

5.5 Recharge Unit

No.	Item	Specification
1	Standard Electricity	100-240 V~, 0.8-0.4 A
2	Voltage	28 VDC
3	Capacity	1.42 A

5.6 Geometry and Mechanical Data

Mechanical Data

No.	Item	Specification
1	SSD	200 mm
2	Out radiation field	Φ 60

Collimator

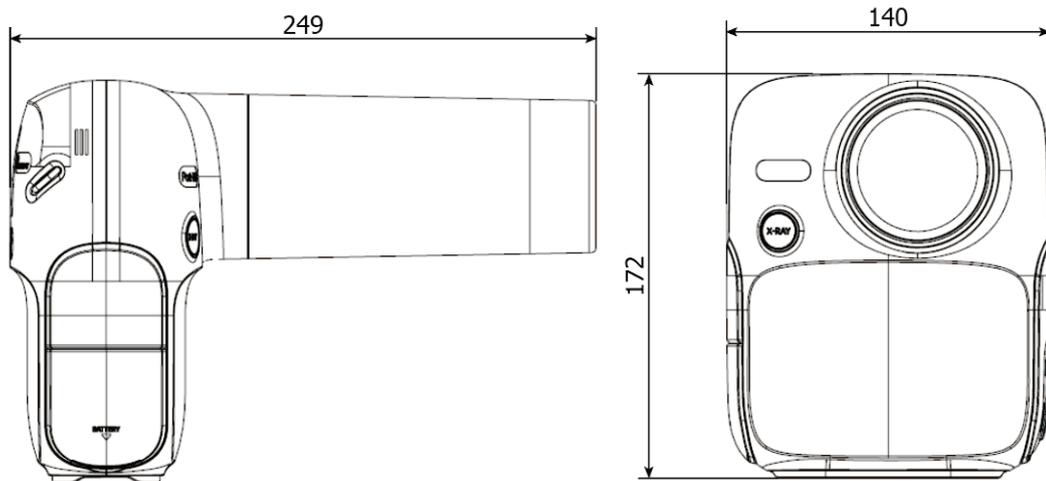
No.	Item	Specification
1	Type	Cylinder
2	Shield Material	2.0 mm Pb

5.7 Sensor (Option)

Item	Specification
Scintillator type	CsI(Tl)
Pixel size(μm)	20 x 20
Pixel pitch (HxV) (μm)	20
Number of effective pixels(px)	1000 x 1500 / 1300 x 1700
High resolution	20 Lp/mm
Dynamic Range	57 dB
Interface	USB 2.0

5.8 Dimensions and weight

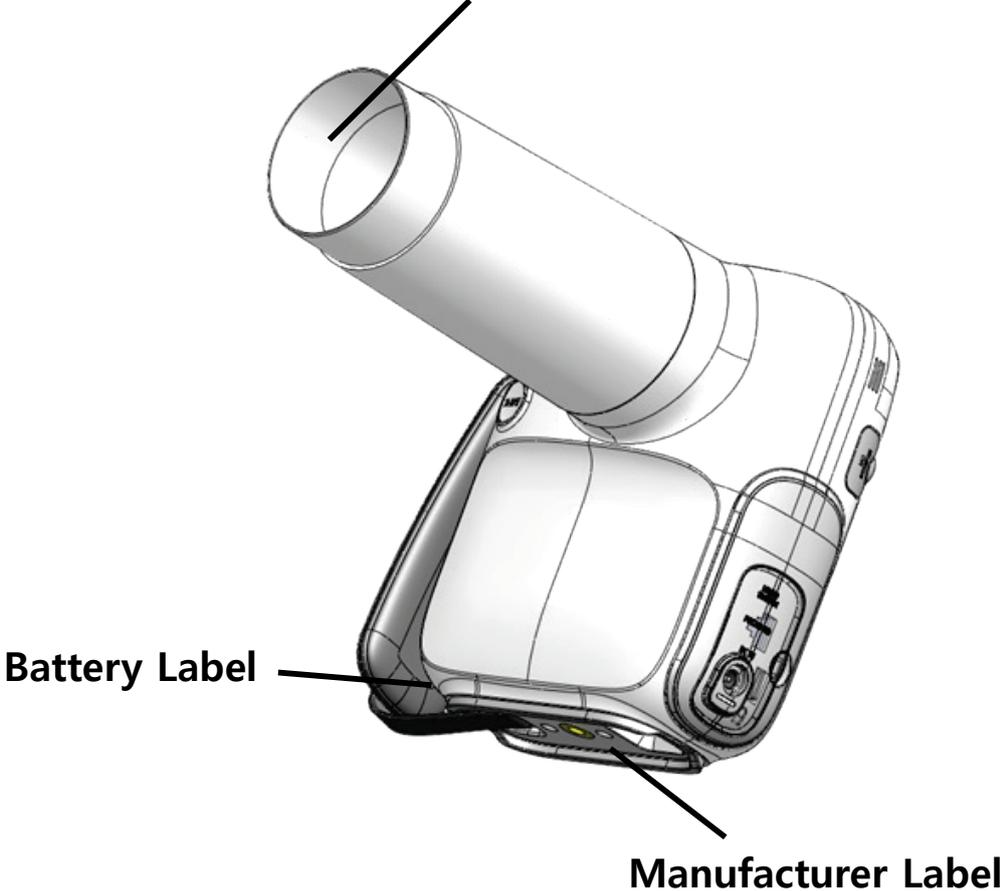
No.	Item	Specification	
1	Dimension (Unit: mm) ±10 mm	Width	140
		Hight	172
		Depth	249
2	Weight	1.5 kg±5 %	



5.9 Equipment Labeling

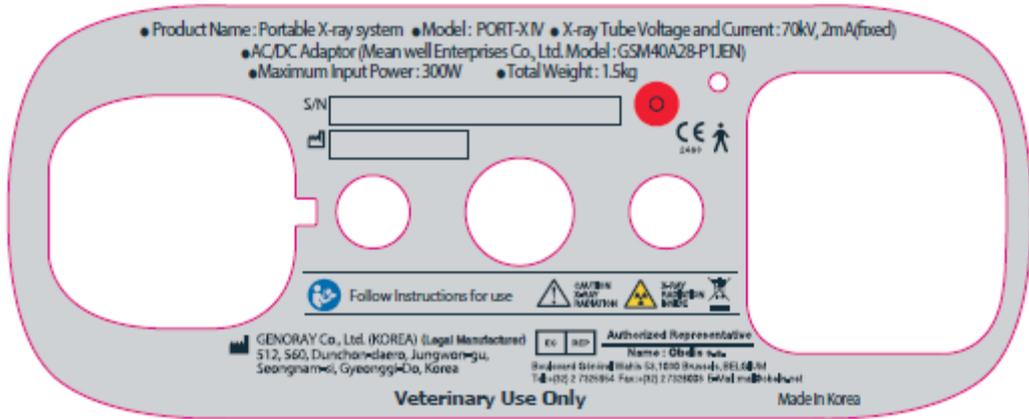
Be sure to check the label and manual before turning on the power.

X-ray Tube Assembly Label

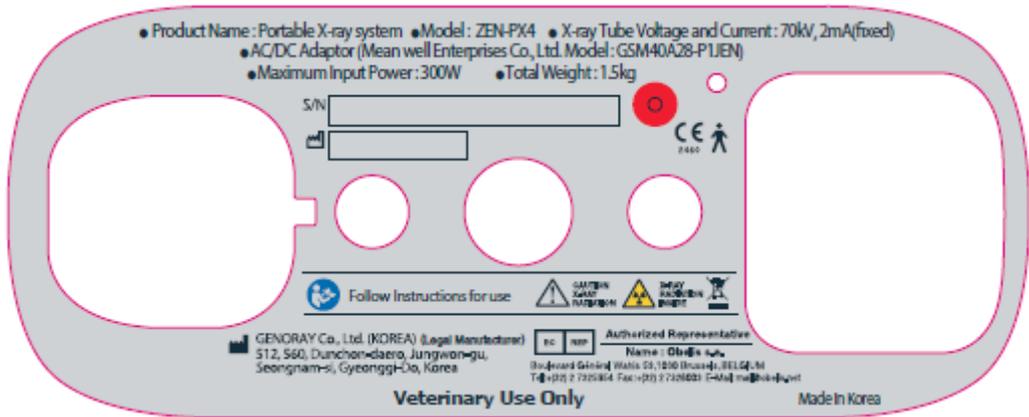


Manufacturer display items

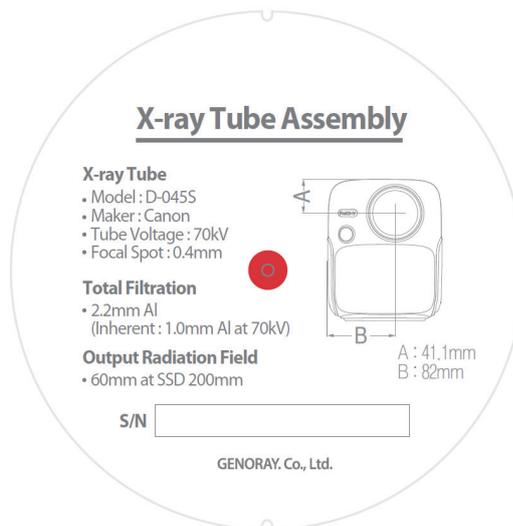
- PORT-X IV



- Multiple name



Tube label



Battery

GENORAY

Product : Rechargeable Li-ion Battery Pack
Type of battery : 6ICP6/35/62
Model : GPB-1100
Rating(s) : 22.2 VDC | 1000mAh
Distributor : GENORAY Co., Ltd.
Manufacturer : GENORAY Co., Ltd. (Made in Korea)
S/N

 **SAFETY WARNING** 

1. Use only the charger designated for this battery pack.
2. Store this product at room temperature and avoid crush, high temperature and humidity.
3. Disassemble, open, or misuse may present a risk of fire or explosion.

GENORAY

Product : Rechargeable Li-ion Battery Pack
Type of battery : 6ICP6/35/62
Model : P4-6ICP6/35/60_Pc
Rating(s) : 22.2 VDC | 1000mAh
Distributor : GENORAY Co., Ltd.
Manufacturer : BN TECH Co., Ltd. (Made in Korea)
S/N

GENORAY

Product : Rechargeable Li-ion Battery Pack
Type of battery : 6ICP6/35/62
Model : HS603560H25-061PBH
Rating(s) : 22.2 VDC | 1000mAh
Distributor : GENORAY Co., Ltd.
Manufacturer : Han Sung A-Tech (Made in Korea)
S/N

Appendix1. Error Messages and Interim Measures

If a problem occurs while using the equipment and/or is not functioning properly, an error message is displayed on the X-ray control unit.

After checking the message, take appropriate action.

If the same issue is repeated even after taking the temporary measures, contact the service center.

A.1.1 Error message

1	Code number	Error 1
	Error message	OCP ERROR
	Measures	<ul style="list-style-type: none"> ▪ After rebooting the device, check to see if the issue has been resolved. ▪ If the same issue continues, contact the service center.
2	Code number	Error 2
	Error message	The kV is high.
	Measures	<ul style="list-style-type: none"> ▪ After rebooting the device, check to see if the issue has been resolved. ▪ If the same issue continues, contact the service center.
3	Code number	Error 3
	Error message	The kV is low.
	Measures	<ul style="list-style-type: none"> ▪ After rebooting the device, check to see if the issue has been resolved. ▪ If the same issue continues, contact the service center.
4	Code number	Error 4
	Error message	The mA is high.
	Measures	<ul style="list-style-type: none"> ▪ After rebooting the device, check to see if the issue has been resolved. ▪ If the same issue continues, contact the service center.
5	Code number	Error 5
	Error message	The mA is low.
	Measures	<ul style="list-style-type: none"> ▪ After rebooting the device, check to see if the issue has been resolved. ▪ If the same issue continues, contact the service center.
6	Code number	Error 6
	Error message	The mA is high.
	Measures	<ul style="list-style-type: none"> ▪ After rebooting the device, check to see if the issue has been resolved. ▪ If the same issue continues, contact the service center.

7	Code number	Error 7
	Error message	The tank temperature is high.
	Measures	<ul style="list-style-type: none"> ▪ Please use the device after keeping the power OFF for 30 minutes or more. ▪ If the same issue continues, contact the service center.
8	Code number	Error 8
	Error message	The tank temperature is low.
	Measures	<ul style="list-style-type: none"> ▪ Use after leaving it in a warm place for 30 minutes or more. ▪ If the same issue continues, contact the service center.
9	Code number	Error 9
	Error message	One of the kV/mA output is high.
	Measures	<ul style="list-style-type: none"> ▪ After rebooting the device, check the symptoms. ▪ If the same issue continues, contact the service center.
10	Code number	Error 10
	Error message	The shot switch is broken or depressed.
	Measures	<ul style="list-style-type: none"> ▪ Release the Hand switch or Shot switch after the shot. ▪ If the same issue continues, contact the service center.
11	Code number	Error 11
	Error message	Image creation failure.
	Measures	<ul style="list-style-type: none"> ▪ After reconnecting the oral sensor, take the image again. ▪ If the same issue continues, contact the service center.
12	Code number	Error 12
	Error message	Temp sensor malfunction.
	Measures	<ul style="list-style-type: none"> ▪ Apply for service. ▪ If the same issue continues, contact the service center.

A.1.2 Warning Message

1	Code number	Warning 00
	Error message	X-ray exposure is limited while the battery is charging.
	Measures	<ul style="list-style-type: none"> ▪ Use after charging is completed. ▪ If the same issue continues, contact the service center.
2	Code number	Warning 01
	Error message	Low battery level
	Measures	<ul style="list-style-type: none"> ▪ Use after charging the battery. ▪ If the same issue continues, contact the service center.
3	Code number	Warning 02
	Error message	FFC file is missing.
	Measures	<ul style="list-style-type: none"> ▪ Please create a FFC file.
4	Code number	Warning 03
	Error message	Low backup battery level.
	Measures	<ul style="list-style-type: none"> ▪ Replace the backup battery. ▪ If the same issue continues, contact the service center.
5	Code number	Warning 04
	Error message	Time synchronization failed.
	Measures	<ul style="list-style-type: none"> ▪ Please retry time synchronization. ▪ If the same issue continues, contact the service center.
6	Code number	Warning 05
	Error message	Wireless dongle connection failed.
	Measures	<ul style="list-style-type: none"> ▪ Please reconnect the wireless dongle. ▪ If the same issue continues, contact the service center.
7	Code number	Warning 06
	Error message	Server PC connection failure.
	Measures	<ul style="list-style-type: none"> ▪ Check the Wi-Fi connection and the server PC settings. ▪ If the same issue continues, contact the service center.
8	Code number	Warning 07
	Error message	Image transmission failed.
	Measures	<ul style="list-style-type: none"> ▪ Try sending the image again. ▪ If the same issue continues, contact the service center.

9	Code number	Warning 08
	Error message	Micro SD card connection failure.
	Measures	<ul style="list-style-type: none">▪ Check the micro SD card connection.▪ If the same issue continues, contact the service center.
10	Code number	Warning 09
	Error message	Oral sensor connection failure.
	Measures	<ul style="list-style-type: none">▪ Reconnect the oral sensor.▪ If the same issue continues, contact the service center.

Appendix2. X-ray Exposure settings



X-ray Exposure conditions are recommended irradiation conditions when using digital sensors. The Exposure conditions may vary depending on the specifications and performance of the digital sensor used by the user.

A.2.1 Initial setting

Digital Sensor

Patient Type	Exposure time (sec)		
	Tooth 1	Tooth 2	Tooth 3
Large	0.63	0.63	0.56
Medium	0.50	0.50	0.45
Small	0.40	0.40	0.36

Appendix3. Air-kerma & Dose Information

Step	Sec	Air-kerma (mGy)	DAP (mGy*cm ²)	Step	sec	Air-kerma (mGy)	DAP (mGy*cm ²)
5	0.05	0.13	3.65	20	0.32	0.83	23.34
6	0.06	0.15	4.38	21	0.36	0.93	26.26
7	0.07	0.18	5.11	22	0.40	1.03	29.18
8	0.08	0.21	5.84	23	0.45	1.16	32.82
9	0.09	0.23	6.56	24	0.50	1.29	36.47
10	0.10	0.26	7.29	25	0.56	1.44	40.85
11	0.11	0.28	8.02	26	0.63	1.63	45.95
12	0.13	0.34	9.48	27	0.71	1.83	51.79
13	0.14	0.36	10.21	28	0.80	2.06	58.36
14	0.16	0.41	11.67	29	0.90	2.32	65.65
15	0.18	0.46	13.13	30	1.00	2.58	72.94
16	0.20	0.52	14.59	31	1.12	2.89	81.70
17	0.22	0.57	16.05	32	1.25	3.22	91.18
18	0.25	0.64	18.24	33	1.40	3.61	102.12
19	0.28	0.72	20.42	34	1.60	4.13	116.71

- DAP value was calculated considering the EXIT FIELD SIZE ($\Phi 60$ mm) with the estimated air kerma value. (Formula for DAP = Air-kerma * $\Phi 60$ mm)
- D.A.P (Dose Area Product) Model : Kerma X plus DDP "single"



The error range for the measured dose value is within $\pm 10\%$ on average.

Appendix4. Maintenance

Experienced users are responsible for the proper operation and maintenance of the system.

Check regularly for the safety of the patient and the operator as per the following information.

A.4.1 Cleaning

- Use a disinfectant (less than 17% alcohol) with a cloth to clean the PORT-X IV.

Always keep the tip of the cone clean. This portion of the unit is in frequent contact with the patients.

- Be careful not to let the cleaning spray solution or disinfectant, etc. enter into the equipment.

This may cause electric failure.

- Do not use cleaners meant for hard surfaces.

Certain chemical combinations can degrade plastics.

- Do not use cleaning agents containing solvents that can dissolve plastics.



Before cleaning the equipment, be sure to turn off the power and disconnect the power plug from the outlet.

A.4.2 Dose Confirm

The PORT-X IV's X-ray generating is factory calibrated and tested.

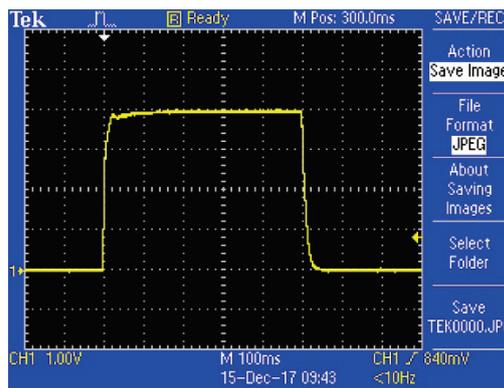
The following checks are to be performed by a qualified technician.

The test items measure the X-ray tube voltage/current, radiation time and dose and can diagnose the condition of the equipment with the measured results.

For measurement instruments, we recommend that you use certified instruments from a specialist supplier.

Measurement Method

- Tube current measurements are performed using an oscilloscope.
- Connect the oscilloscope to no. 6 and no. 8 of the RJ45 external output terminal on the equipment.
- Exposure time from RJ45 No. 6 (mA_OUT) to RJ45 No. 8 (GND) is set at 0.5 sec and then the X-ray is exposed.
 - Reference value: 2 mA -> 4 V \pm 10% (3.6 to 4.4 V)



mA: 0.5 mA per 1 V

- If the oscilloscope's waveform is not displayed correctly then take another x-ray exposure.
- If the output waveform does not match the reference value, contact the manufacturer.
- Tube voltage, dose value and exposure time measurements are carried out using a dosimeter.

Set the exposure time (refer to Appendix. 2) as follows and exposure X-rays.

Item	Reference value	Timer setting and its allowable range				
		50 ms	100 ms	500 ms	1000 ms	1600 ms
kVp (eff) Accuracy	70 kV \pm 7%	66-75	66-75	66-75	66-75	66-75
Timer Accuracy	Set point \pm 10%, \pm 20 ms	45-55	90-110	450-550	900-1100	1440-1760

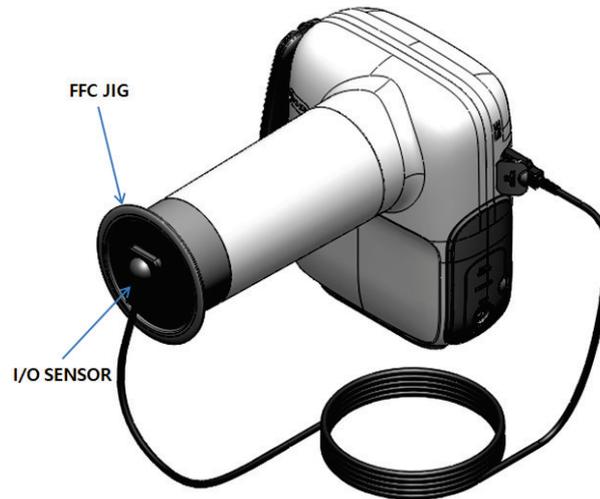
- If it is out of the above error range then take another x-ray exposure.
If it is continuously out of the error range, contact the manufacturer.
- Details regarding the measuring equipments used are as follows.
We recommend using the following equipment or equivalent certified products.

Model	Manufacturer	Photo
Oscilloscope (TDS2022C)	Tektronix	
Dose Meter (255)	Piranha	

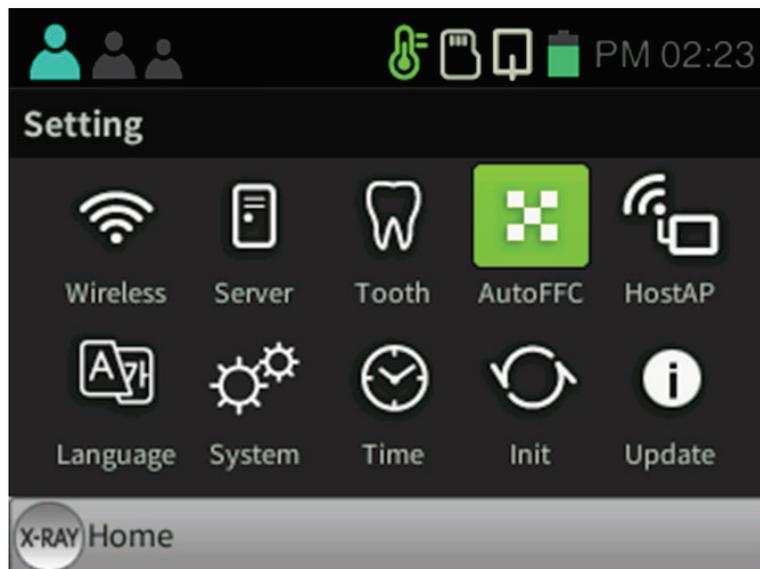
A.4.3 Check image

Please refer to "Dose Confirm" item to check whether the X-ray output is normal.

Make Flat Field Correction



In order to obtain a low-noise image, execute the FFC (flat filed correction) function by using the FFC JIG provided for the first connection with the PORT-X IV and Intra Oral sensor

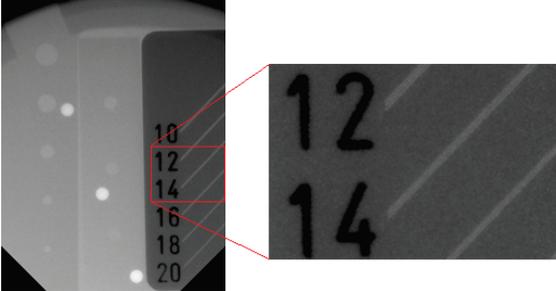


Select Setting-AutoFFC to execute the FFC function automatically.

Resolution

Check the image quality by measuring the clarity of the image via resolution.

Please see instructions below:

Logistics	Photo
<p>① Attach the I/O Sensor to the phantom.</p>	
<p>② Exposure X-ray for 0.2 seconds.</p>	
<p>③ The lines of the resolution chart should appear as shown in the image.</p> <ul style="list-style-type: none"> • More than 14 Line Pairs should appear. • If the line pair is not visible, proceed to FFC and take another exposure. <p>If the line pair is still not visible, please contact our service center.</p>	

- Additional information on the phantom:

Model	Manufacturer	Photo
<p>QUART dent/digitest M2/M1</p>	<p>QUART</p>	

A.4.4 Service

- Only technicians certified by Genoray Co. can perform inspection and service of equipment.
- For technical support, please contact our service center.

A.4.5 Returns and repairs

- If the equipment is found to be defective and needs repair, please contact our service center.
- Genoray Co. will not be responsible if the product is disassembled or modified arbitrarily.

A.4.6 Maintenance table

Inspection period	Inspection items	Inspector	Result					Remarks
			1	2	3	4	5	
Everyday	Power plug connection status	User						
	Grounding connection status							
	Power off after using the equipment							
	Cleanliness of the equipment							
	Disinfecting sections that come into contact with patient							
	On/off switch							
	Up/down, right/left buttons							
	Exposure Switch							
	Patient registration							
	Saving and loading images							
Every week	Temperature and humidity							
	Emergency switch							
6 months or more Unused equipment	Tube voltage accuracy test	Manufacturer						
	Tube current accuracy test							
	Exposure dose to the patient							
	Detector calibration							
	Charge the battery							



Carry out periodic inspections and tests to maintain great equipment and image quality.



For a longer life expectancy of the unit, regular inspections must be made every 3 years after the date of purchase.



 GENORAY Co., Ltd. (Legal manufacturer)

512, 560, Dunchon-daero, Jungwon-gu,
Seongnam-si, Gyeonggi-do, Republic of Korea

Tel. +82-31-5178-5500

Fax. +82-31-5178-5598

Email. genoray@genoray.com

Web. www.genoray.com



Obelis s.a.

Boulevard Géznéral Wahis 53, 1030 Brussels, BELGIUM

Tel. +32-2-732-59-54 Fax. +32-2-732-60-03 Email. mail@obelis.net

